Körperanalysewaage Barbara

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Dear Customer!

You have chosen to purchase a high-quality ADE brand product, which combines intelligent functions with an exceptional design. This set of body analyser scales enables you to measure and get to know yourself and your body better.

The wealth of experience of the ADE brand ensures a high technical standard and proven quality.

We wish you much enjoyment while weighing yourself and discovering your values!

Your ADE Team

General information

About this operating manual

This operating manual describes how to operate and maintain your product safely. Keep this operating manual in a safe place for future reference. If you pass this product on to someone else, pass on this operating manual as well.

Failure to observe this operating manual may result in injuries or damage to your product.

Explanation of symbols

This symbol combined with the word DANGER warns of serious injuries.

This symbol combined with the word WARNING warns of moderate and mild injuries.

This symbol combined with the word NOTE warns of material damage.

This symbol indicates additional information and general notes.

Table of Contents

General information	16
Intended use	18
Safety	18
Overview	20
Scope of delivery	20
Start-up	21
This is how your body analyser scales work.....	21
Measuring your weight	22
Entering personal data	22
Body analysis.....	23
Changing the battery.....	26
Cleaning	26
Fault/Remedy.....	26
Technical data.....	27
Declaration of conformity	28
Warranty	28
Disposal.....	28

Bamboo is a natural material that is alive and changes over time. Therefore, slight surface cracking can occur over time.

Intended use

The bathroom scale is designed for the weighing of people in the home. It is not suitable for use in clinics, medical practices, etc. Likewise, the body analysis function is designed only for private use. The scales are not suitable for use in the professional sporting sphere.

The maximum carrying capacity of 150 kg must not be exceeded.

Do not use these scales if you ...

- ... are pregnant,
- ... have a cardiac pacemaker,
- ... are a dialysis patient,
- ... suffer from oedema,
- ... have implants, e.g. artificial limbs, metal plates, etc.

If in doubt, consult your GP.

Safety

- This product may be used by children aged 8 and above, as well as by persons with reduced physical, sensory, or mental capacities or lacking experience and knowledge if they are supervised or have been instructed in the safe use of the product and understand the risks that may arise.
- Children must not play with the product.
- Keep packaging material away from children. There is a risk of choking if swallowed. Also pay attention that children do not pull the packaging material over their heads.
- Cleaning and user maintenance must not be carried out by children, unless they are 8 years of age or older and are being supervised.
- Batteries can be life-threatening. Therefore, keep batteries out of reach of small children. If a battery has been swallowed or has entered the body in any other way, seek medical help immediately.
Failure to do so within 2 hours may result in severe internal burns that could cause death.

RISK of fire/burning and/or explosion

- There is a risk of explosion if the batteries are replaced improperly. Replace the battery using only the same or an equivalent type of battery. Pay attention to the “Technical data”.
- Batteries must not be recharged, reactivated by other means, dismantled, thrown into fire, or short-circuited.
- Avoid contact with skin, eyes, and mucosa. In the event of contact with battery fluid, rinse the affected areas immediately with plenty of clean water and seek medical attention immediately.

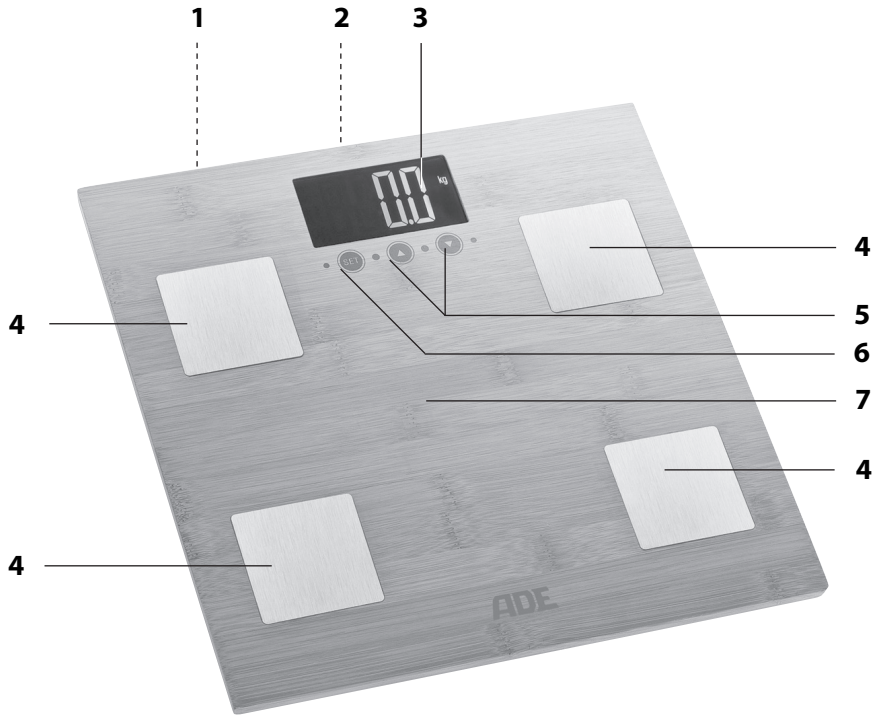
WARNING of risk of injury

- Do not step onto the outer edge of any one side of the scale. Risk of tipping!
- Do not step onto the scale with wet feet or when its surface is damp. Risk of slipping!

BEWARE of material damage

- Place the scale on a stable, even surface.
- Do not overload the scale. Pay attention to the “Technical data”. Place yourself on the scale exclusively for weighing purposes.
- Protect the scale from strong shocks and vibrations.
- Remove the batteries from the scales if they have been used up or if you are not going to use the scales for a long time. You will thus avoid damage that may result from leakage of battery acid.
- When replacing the batteries, pay attention to the correct polarity (+/-).
- Do not subject the batteries to any extreme conditions, by – for example – storing them on radiators or under direct sunlight. Increased risk of leakage!
- Clean the battery and device contacts, as needed, before inserting the battery.
- Do not make any modifications to the product. Only allow repairs to be carried out by a specialist workshop. Repairs carried out improperly can considerably endanger the user.
- There are feet on the underside of the scale. Since floor surfaces are made from various materials and are treated with diverse care products, it cannot be completely excluded that some of these substances contain ingredients that may adversely affect and soften the feet of the device. If necessary, place a non-slip pad under the scale.

Overview

- 1 UNIT button (back): Changing the unit of measurement
- 2 Battery compartment (reverse side)
- 3 Display
- 4 Sensor
- 5 ▲▼ buttons: Select memory cell, reduce data, increase data
- 6 SET button: Call up setting mode, save inputs, switch scale to body analysis mode
- 7 Weighing surface

Scope of delivery

- Body Analysis Scales
- 2x CR2032 batteries, 3 V --- (pre-inserted)
- Operating Manual

Start-up

Removing iso-strips

The batteries are already inserted on delivery and, for protection against premature discharge, are provided with an iso-strip. In order to start up the scale, you must remove the iso-strip.

The battery compartment is located on the underside of the device.

1. Push the locking tab towards the battery compartment cover and remove the battery compartment cover.
2. Pull out the iso-strip.
3. Replace the battery compartment cover.

Choosing the unit of measurement

When the scales are delivered, the unit of measurement is set to “kg” as standard. The measurement result can also be displayed in English units “lb” (pound) and “st lb” (stone pound).

1. To switch on the scale, exert a little force on the weighing surface.
2. To select the desired unit of measurement, press repeatedly the UNIT button on the back of the scale. Note the display.

Setting up the scale

- Place the scale on an even, firm and dry surface. Do not place the scale on a carpet.

This is how your body analyser scales work

Your new body analyser scales work using so-called BIA-technology (Bioelectric Impedance Analysis). This involves an undetectable, completely harmless and weak current flowing through your body.

The measurement takes place via sensors that are mounted on the weighing surface.

- The current flows relatively unhindered through the fluid contained in muscle mass.
- In fat mass, the resistance is greater. This resistance is measured.

Together with your personal data (sex, weight, age, and height), the scales calculate the following information about your body composition:

- Weight
- Body fat proportion in %
- Body water in %
- Muscle mass in %
- Bone mass in %

The scale has a total of 10 memory cells.

Measurement errors

Measurement or calculation errors may arise for the following groups of people during the body analysis:

People, who ...

... spend 10 hours or more per week playing sport,

... are professional athletes, body builders or endurance athletes,

... are over 70 years old,

... are less than 10 years old.

The “normal weighing function” (see next chapter) is not affected by these considerations.

Measuring your weight

For normal weighing – without body analysis – you do not have to input any data.

- Stand still on the scale. Stand still and do not hold on to anything.

The measured weight will be displayed shortly thereafter.

A few seconds after you step off the weighing surface, the scale will switch off automatically.

Entering personal data

1. To switch on the scale, step on or briefly press once down firmly on the weighing surface.
“0.0” and the set unit of measurement, e.g. “kg”, will be shown on the display.
2. Press SET.
“P1” (P1-P10) flashes on the display.
3. Use the ▲ or ▼ button to select the desired memory cell.

4. Press SET.
Use the ▲ or ▼ button to select your gender:  for male, or  for female.
5. Press SET.
On the display, the preset value for age (preset: 30).
6. Use the ▲ or ▼ button to enter your age.
7. Press SET.
The preset body height value (e.g. "170" for cm) flashes on the display.
8. Use the ▲ or ▼ button to enter your height.
9. Finally, press SET. You have thus input the necessary data. All inputs are then displayed all together once more.
The display then shows "0.0" and you can proceed with your body analysis.
If you do nothing else, the scales will automatically switch off after a few seconds.

- Keeping the ▲ or ▼ button pressed displays the data faster in forward and reverse direction, respectively.
- Entered data can only be overwritten by making a new entry.
- If the time interval between two pushes of a button is more than 10 seconds, the scale will switch off and you should repeat the procedure.

Body analysis

- Body analysis is only possible if your data has been entered (see previous chapter).
- See the chapter "Weighing Tips".

1. Press SET to switch on the scale.
2. Use the ▲ or ▼ button to select the memory cell where your personal body data are stored.
3. Press SET repeatedly to confirm your displayed data successively.
The data are then briefly displayed in succession. The display then shows "0.0" and the selected unit of measurement, e.g. "kg".
4. Position yourself barefoot on the scales. Stand still and do not hold on to anything.
 - In doing so, ensure each foot is placed on a sensor.
 - There must be no contact between your feet, your legs, your calves, or your thighs. Otherwise, the measurement will be imprecise.

The following values are displayed one after another:

	Indicator	Meaning
1.		Your body weight in the set unit of measurement
2.	Overfat fat	Your proportion of body fat in %
3.	TBW	Your body water in %
4.		Your muscle mass in %
5.		Your bone mass in the selected unit of measurement, e.g. "kg"

Weighing tips

- Weighing for the body analysis can only be carried out barefoot. In doing so, your feet should only be lightly moistened. Completely dry feet can lead to false results, since the electrical conductivity will then be poor.
- During the measuring process, stand still and do not hold onto anything.
- Do not weigh yourself directly after unusual physical exertion.
- Do not weigh yourself immediately after standing up. Wait approximately fifteen minutes. In this time, the water is able to distribute through your body.
- Always attempt to weigh yourself at the same time, in order to obtain comparable values.

Notes on the ascertained values

The proportion of body fat gives the amount of fat as a percentage of the total weight of the body. A blanket recommendation for this value does not exist, since this depends significantly on sex (because of different physiques) and age.

Body fat is vital for day-to-day body function. It protects organs, cushions joints, regulates body temperature, stores vitamins, and serves the body as an energy store.

For women, a body fat proportion between 10% and 13% is essential; for men, between 2% and 5% with the exception of competitive sportsmen.

The proportion of body fat is not an unambiguous indicator of health. Excessive body weight and a too-high proportion of body fat are being linked with the appearance of many diseases of affluence such as diabetes, cardiovascular illnesses, etc.

Physical inactivity and poor diet are often triggers for these diseases. There is a clear causal connection between excess weight and physical inactivity.

A considerable part of the human body consists of water. This body water is variably distributed throughout the whole body. Lean muscle tissue contains about 75% water, blood around 83% water, body fat approx. 25% water and bones have a water proportion of about 22%.

- In men approx. 60% of body mass consists of water.
- In women, that value lies around 55% (determined by a higher proportion of body fat).

Of this total body water approx. two thirds are stored in the cells and is therefore called intracellular water. The other third is extracellular water.

Women ♀					
Age	Body fat				Body water
	Slender	Normal	Slightly overweight	Overweight	Normal
10-16	< 18%	18-28%	29-35%	> 35%	57-67%
17-39	< 20%	20-32%	33-38%	> 38%	47-57%
40-55	< 23%	23-35%	36-41%	> 41%	42-52%
56-85	< 24%	24-36%	37-42%	> 42%	37-47%

Men ♂					
Age	Body fat				Body water
	Slender	Normal	Slightly overweight	Overweight	Normal
10-16	< 10%	10-18%	19-23%	> 23%	58-72%
17-39	< 12%	12-20%	21-25%	> 25%	53-67%
40-55	< 13%	13-21%	22-26%	> 26%	47-61%
56-85	< 14%	14-22%	23-27%	> 27%	42-56%

There are no general reference values for muscle mass.

Changing the battery

A change of batteries becomes necessary when “Lo” appears on the display, or when nothing at all is displayed.

When changing the battery, bear the following in mind:

- The data that has been ascertained and entered up to that point remains stored on the scales.
- Only use the type of battery that is specified in the “Technical data”.
- Pay attention to the correct polarity (+/-) when inserting the battery. The plus pole (+) must point upwards.
- Dispose of the old battery in an environmentally responsible way (see “Disposal”).

Cleaning

NOTE on material damage

- Do not immerse the scale in water or other liquids.
- Never use harsh, scouring or abrasive cleaning agents or the like to clean the scale. Otherwise, the surface could be scratched.

1. Wipe the scale as needed with a slightly damp soft cloth.
2. Disinfect the weighing surface and the sensors from time to time using a suitable disinfecting agent (available at specialist retailers and pharmacies).

Fault/Remedy

Fault	Possible cause
No function	<ul style="list-style-type: none">▪ Are the batteries empty or inserted the wrong way around?
The scale appears to be displaying wrong weighing results	<ul style="list-style-type: none">▪ Is the scale located on a firm, even surface?▪ Is a device that can cause electromagnetic disturbances located in the vicinity of the scale?▪ Remove the batteries from the scale and wait a moment. Reinsert the batteries. The data stored until then are retained.
“Lo” is shown on the display	<ul style="list-style-type: none">▪ The batteries are empty, see “Changing the battery”.

Fault	Possible cause
"Err" is shown on the display	<ul style="list-style-type: none"> ▪ The maximum capacity of the scale (150 kg) has been exceeded.
"Err2" is displayed	<ol style="list-style-type: none"> 1. Check whether the scale is correctly positioned. 2. Ideally, have an assistant carry out the weighing process once again. If "Err2" is displayed again, the weighing sensors are defective. In this case, contact the dealer where you bought the scale.

Technical data

Model:	BA 914 body analysis scale "Barbara"
Battery:	2x CR2032 / 3 V 
Rated current:	30 mA
Measuring range:	max. 150 kg
Variance:	2% +/-100 g
Indicator division	
Weight and bone mass:	100 g
Body water, muscle mass, body fat:	0.1%
Memory cells:	10
Age input:	10-80 years
Height:	100-240 cm
Scale dimensions:	approx. 300 x 300 x 34 mm
Scale weight:	approx. 1 kg
Distributor: (no service address)	Waagen-Schmitt GmbH Neuer Höltigbaum 15 22143 Hamburg, Germany

Our products are constantly being developed and improved.
For this reason, design and technical modifications are possible at any time.

Declaration of conformity

Waagen-Schmitt GmbH hereby declares that BA 914 body analysis scale complies with EU directives 2014/30/EU and 2011/65/EU.

The declaration will lose its validity if any modification is made to the device without our consent.

Hamburg, March 2021
Waagen-Schmitt GmbH

Warranty

Waagen-Schmitt GmbH warrants for 2 years from the purchase date the cost-free rectification of defects in material and workmanship through repair or exchange.

In the event of a warranty claim, please return the product with the proof of purchase (specifying the reason for claim) to your dealer.

Disposal

Disposing of the packaging

Dispose of the packaging according to type. Dispose of cardboard with waste paper and of films with recyclable materials.

Disposing of the product

Dispose of the product in accordance with the regulations that apply in your country.

Devices must not be disposed of in normal household waste

At the end of its life, the product must be disposed of in an appropriate way. In this way, valuable materials contained in the device will be recycled and the burden on the environment avoided. Bring the old device to a collection point for electronic waste or to a recycling depot.

For more information, contact your local waste disposal firm or your local administrative body.

Standard and rechargeable batteries should not be disposed of along with household waste

All batteries and rechargeable batteries must be brought to a local or district collection point or to the retailer. Such batteries and rechargeable batteries can thus be disposed of in an environmentally friendly way.

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