## AFH TheraPIE Sand® Sensorik

DE

EN

FR

ES

IT

NL

**DE**

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## AFH TheraPIE Sand® Sensorik

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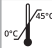
# Instructions for use

## AFH TheraPIE Sand® Sensorik

	Follow instructions for use	<b>MD</b>	Medical device
	Application only for one patient		Before and after use wash your hands
	Not for children under 3 years		Not suitable for eating
	Protect from moisture		Protect from sunlight
	Latex free		Permissible temperature range

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**Intended use and areas of application:**

This medical device is latex-free AFH TheraPIE Sand® Sensorik, which uses special finger and hand exercises to improve the movement and function of the neuromuscular control circuit. It promotes sensitivity in the fingers, hand and ankles. Stimulates blood circulation and promotes stress reduction.

AFH TheraPIE Sand® Sensorik is primarily used in therapy to promote sensitivity and optimize the mobility of finger, hand and ankle joints. It improves general hand strength and hand and foot coordination. The AFH TheraPIE Sand® Sensorik is also an excellent educational tool, especially in the fields of occupational therapy and physiotherapy, for the purpose of stimulating and developing or promoting creativity.

If the ability to move the fingers, hands and feet with their associated muscles is restricted or made difficult by pain, the AFH TheraPIE Sand® Sensorik can be used successfully to alleviate and improve the current situation. AFH TheraPIE Sand® Sensorik significantly facilitates the gradual mobilization of a wide range of restrictions in the area of finger, hand and foot motor skills, especially their sensory functions.

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Due to its special kinetic and fluffy properties, AFH TheraPIE Sand® Sensorik is suitable for improving fine and gross motor skills in the finger, wrist and ankle joints. The kinetic properties promote the joy of movement. The flow properties optimize and help to improve sensitivity. Everyday stress is reduced. This makes it an excellent medium for stress management. With these properties, stress conditions are reduced re-

spectively the therapy sand helps to reactivate concentration. It is also an ideal medium for general strengthening of the finger, hand and foot muscles. situation. AFH TheraPIE Sand® Sensorik significantly facilitates the gradual mobilization of a wide range of restrictions in the area of finger, hand and foot motor skills, especially their sensory functions.

### **Contraindications:**

Diseases of the skin in the palm area and foot area (e.g.: neurodermatitis, open wounds).

### **Risks of use / Important information / Properties:**

- Use AFH TheraPIE Sand® Sensorik according to indication.
- AFH TheraPIE Sand® Sensorik is available in the sizes 454 g and 2 kg in 5 different colors with the same properties / resistance. The choice of color depends on personal preference, but can also be combined with different colors.
- The quantity depends on the exercise purpose or training program to be carried out (e.g. 454g to 2kg for fingertip grip exercises of the wrists and ankles through to foot muscle training in, for example, a suitable tub)
- Simultaneous use with other products only after consultation with your doctor or therapist.
- Do not make any changes to the product.
- Do not use with open wounds in the hand and foot area.
- Do not reuse - this aid is intended for the care of one patient.
- Caution: Not suitable for eating.
- Not suitable for children under 3 years of age.
- Wash hands before and after use.
- Do not get on clothing or other textiles.
- If Therapy Sand Sensorik gets into the eye, wash the eye with water. Consult a doctor if necessary.
- If pain, redness, swelling or other problems occur, reduce the duration of the exercise frequency or stop the entire exercise program.

Consult your therapist, prescribing doctor or other medical professional about use and duration. Instruction is provided by the specialist personnel already listed from whom you received the medical device.

## Sensors Exercises Application examples 454 g

Pointed grip 01

Pointed grip 02

3-point grip

Spherical handle

Sand trickle one-handed

Shaping balls

## Sensors Exercises Application examples 2 kg

Finding marbles 01

Finding marbles 02

Sand trickling bilateral with tub

Sand trickling / gripping force with toes 02

Finding marble / grasping with toes 01

Finding marbles / gripping with toes 02

**Focus:**

Excellent motor training for finger, hand and foot mobility. In addition, the AFH TheraPIE Sand® Sensorik supports the regeneration of sensitivity and sensory abilities. With regular use, you can improve general muscular strength, increase dexterity and mobility, develop motor skills and help to optimize blood circulation. It is also recommended for all forms of stress management. The AFH TheraPIE Sand® Sensorik is available in 5 colors with the same properties.

**Further exercises are possible depending on the therapeutic instructions.****Storage:**

Due to the viscous or kinetic flow behavior of AFH TheraPIE Sand® Sensorik and to avoid contamination, AFH TheraPIE Sand® Sensorik should always be stored in an airtight, sealed bag in the supplied container at room temperature.

**Material composition:**

Quartz sand (82%), glass beads (6.8%), methylvinyl-polysiloxane (4.5%), polydimethylsiloxane (2.5%), calcium carbonate (2.3%), magnesium oxide (1.3%), borax (0.1%), color pigments (0.5%).

**Cleaning instructions:**

The therapy sand cannot be cleaned. Please wash and disinfect your hands thoroughly before and after use.

**Warranty:**

The statutory provisions of the country in which you purchased the product apply to the purchased product. Please contact your specialist dealer if you suspect a warranty claim. Please clean the product before submitting a warranty claim. If the enclosed instructions for use have not been sufficiently observed, the warranty may be impaired or invalidated. Warranty is excluded in the event of improper use, failure to observe the risks and instructions for use and unauthorized modifications to the product.

**Service life / service life of the product:**

The service life of the AFH TheraPIE Sand® Sensorik is determined by natural wear-and-tear when handled properly and in accordance with the application. At the end of treatment, therapy or customized training, the AFH TheraPIE Sand® Sensorik must be disposed of properly.

**Obligation to report:**

If there is a serious deterioration in the state of health when using the medical device, please report this to your specialist dealer or to us as the manufacturer and to the BfArM (Federal Institute for Pharmaceuticals and Medical Devices).

If you are resident in Switzerland, please contact our Swiss authorized representative at: [info@fg-health.ch](mailto:info@fg-health.ch) or directly to Swissmedic.

**Disposal:**

At the end of its service life, the product must be disposed of in accordance with local regulations.

**Declaration of conformity:**

We confirm that this product complies with the requirements of REGULATION (EU) 2017/745 of the European Parliament and of the Council.

---

## AFH TheraPIE Sand® Sensorik

The AFH TheraPIE Sand® Sensorik is available in the sizes 454 g and 2 kg and in 5 colors with the same properties / resistance.

colors	weight
	454 g / 2 kg

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