# Backright® SEAT CUSHION

English

Thank you for purchasing the Backright® Seat Cushion, which helps you sit comfortably with good posture. It helps prevent:

- back pain
- pain associated with sciatica

It's perfect for people who suffer from haemorrhoids.

Its ergonomic design provides support and comfort while reducing pressure on the tail bone.

Memory foam keeps its shape and is breathable to keep the cushion cool. It's perfect for sitting in your office chair, driving and travelling.

### IMPORTANT:

Because this product is made of new materials, it may have that "new" smell. The smell can be reduced by taking the Backright® Seat Cushion out of the bag to air it. This process may take a few days. To immediately eliminate the "new" smell, we recommend washing the cover. Follow the washing instructions on the label. **CAUTION:** Memory foam reacts to heat. It may seem hard at first, but it will become softer as it absorbs your body heat.

- Keep out of reach of children.
- Prevent the cushion from coming into contact with sharp objects, which could tear the fabric.

• **PRECAUTION:** The non-slip rubber bottom may stain light-coloured leather or vinyl seats. Wash the cover in cold soapy water before use. When the cover is dry, check it for colour fastness with a towel and hot water.

**USE:**  
Place the cushion on the middle of the seat or chair, as shown in Picture 1.  
Position the Backright® Seat Cushion on a flat surface with the U-shaped cut-out pointing towards the backrest, as shown in the picture.

### CLEANING INSTRUCTIONS:

1. Open the zip on the back of the cushion (Picture 2).
2. Put your hand inside the cover and pull on one side of the cushion (Picture 3).
3. Slowly pull the cover off. If the cushion is difficult to remove, fold the memory foam to pull the cover off more easily. (Picture 4)
4. Machine wash cold. **DO NOT PUT MEMORY FOAM IN THE WASHING MACHINE OR DRYER.** (Picture 5)

**COMPOSITION (CUSHION):** 100% memory foam  
**COMPOSITION (CUSHION COVER):** 100% polyester

### AUSTRALIA & NEW ZEALAND ONLY

This product has the benefit of certain Consumer Guarantees. These are prescribed by the Australian Consumer Law & the New Zealand Consumer Guarantees Act 1993 both of which provide protection for consumers. There is no express warranty for this product in Australia or New Zealand. The above paragraph refers to other countries

### Quality Warranty:

This product is covered by a warranty against manufacturing defects subject to the time periods stipulated by the legislation in force in each country.  
This warranty does not cover damage resulting from inadequate use, negligent commercial use, abnormal wear and tear, accidents or improper handling.

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BACKRIGHT SEAT CUSHION® is a registered EU/CTM trade mark. International patents pending. All imitations will be prosecuted.