## DEUTSCH

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## ENGLISH

Dear Customer,

Thank you for choosing this **leg elevation pillow**. If you have any questions, please contact customer service through our website: [www.dspro.de/kundenservice](http://www.dspro.de/kundenservice)

### Explanation of the symbol

Danger symbol: This symbol indicates danger. Please carefully read and follow the associated safety instructions.

### Intended use

- This product is intended to provide back relief.
- The product is **not** a toy!
- The product is intended for private, non-commercial use.
- Use the product only as described in the instruction manual. Any other use is regarded as improper use.

### Safety instructions

- **Risk of suffocation!** Keep packaging materials out of reach of children and animals.
- Never use a compressor or similar high-pressure pumps to inflate the product. This could damage the product.
- Do not sit on the product.
- This product is not suitable for the washing machine or dryer.
- Never use harsh or abrasive cleaners or cleaning pads for cleaning. These can damage the surface.

- Keep the product away from spikes and sharp objects. Do not stick safety pins, needles or other sharp objects into the product.
- Protect the product from heat sources, open flames, sub-zero temperatures, direct sunlight and prolonged moisture.

### Included in delivery

1 x inflatable leg elevation pillow  
1 x foot-operated air pump with 2 valves  
1 x Instructions for use

Check the scope of delivery for completeness, and the components for transport damage. If the product is damaged, do not use but contact the customer service.

### Usage

1. Inflate the leg elevation pillow using the foot-operated air pump:
  - attach one of the valves to the hose
  - connect the hose to the blue nozzle on the foot-operated air pump
  - connect the foot-operated air pump to the valve on the bottom of the product
  - using your foot, inflate the leg elevation pillow until it has reached the desired level of firmness
2. Close the valve cap.
3. Place the leg elevation pillow on the bed or floor.
4. Lying down, place your legs on top of the leg elevation pillow.  
The sides of the leg elevation pillow have different lengths. Choose the side that best suits the length of your legs.

### Cleaning and storage

- Clean the product as required with a soft, damp cloth and mild detergent if necessary. Dry with a soft, clean cloth.
- Ensure that the product is completely dry before deflating and storing it.
- The foot-operated air pump can be used to deflate the product:
  - attach one of the valves to the hose
  - connect the hose to the red nozzle on the foot-operated air pump
  - using your foot, pump the air out of the leg elevation pillow until it is completely empty
- Store the product in a dry place that is protected from the sun and away from children and animals.

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Alle Rechte vorbehalten. | All rights reserved. | Tous droits réservés. | Alle rechten voorbehouden.