Farbige Kontaktlinsen - Deutsch

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Coloured Contact Lenses - English

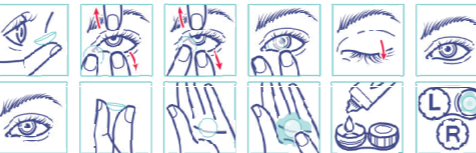
Inserting and Removing your Soft Contact Lenses

Please read these instructions carefully before using your Contact lenses, and we recommend consulting an eye care practitioner, who will be happy to show you how to insert and remove your cosmetic lenses and look after them correctly.

General information:

- Always wash your hands with a mild soap and dry with a lint-free towel before handling your lenses.
- Clean, rinse and disinfect your lenses each time you remove them.
- Always handle the same lens first to avoid mixing up the right and left lenses.
- Remember to handle your lenses gently.

Inserting the lens into your eyes

1. You may want to rinse your lenses in a multi-purpose solution before you put them on.
2. Put one of the lens on the tip of your index finger. Be sure the lens

- is right-side-out (the edges should turn up and not out).
3. Look straight ahead. Pull down your lower eyelid with your middle finger of the same hand.
4. Pull the top lid up with the index finger of the other hand and look up.
5. Gently place the lens on the lower white of the eye.
6. Remove your index finger and release your eyelids.
7. Look down, and close your eyes for a moment. The lens will center itself.
8. Repeat for the other lens.

Removing the lens from your eye

1. You may want to use a lubricating and rewetting drops just before you remove your lenses. One or two drops in each eye will moisten the lenses and make them easier to remove.
2. Look up and pull the lower lid down with you middle finger.
3. Then place your index finger on the lower edge of your lens, and slide the lens down to the white of your eye.
4. Squeeze the lens lightly between your index finger and thumb and remove gently.
5. Repeat for the other eye.
6. If the edges of you lens stick together, place a few drops of saline or multi-purpose solution on the lens and rub gently until the edges separate.

These tips will help guide you, but if you have any eye irritation or a condition that requires attention, please consult your eye care practitioner immediately !

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