
Waterproof Digital Hearing Amplifier
EN Instruction manual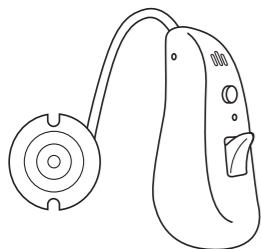

Pro Medix

PR-420

Please read this user manual thoroughly and completely before using the product.

01

Intended use

- This hearing amplifier is only to be used to help people improve their hearing ability.
- This device is only intended for the purpose described in these instructions of use.
- The manufacturer is not liable for damage resulting from improper or careless use.
- This device is an amplifying device and is not a therapeutic medical product.
- This device can neither prevent nor alleviate organically caused hearing loss.

Warning and Safety notice ⚠

- This device is NOT recommended for use by those suffering from Tympanitis or Ear Pyogenesis
- In a case of discomfort and/or pain, please stop using this device immediately and contact your doctor.
- NOT to be used at max volume; a sustained period may cause discomfort.
- This device can NOT be used by infants under 36 months of age.

01


- Keep this device away from children and pets.
- NOT to be used in high temperatures, very humid environments and/or around explosions.
- Avoid contact with magnets near this device; could result in permanent damage (of the device).
- Avoid blockage of ear plug and microphone holes. May result in defective use.
- Clean and maintain the device to ensure optimal use.
- Remove the battery from the device when not being used for long periods of time.
- Remove the sealing sticker on the battery before using. The battery will begin discharging in about 1 minute once the sealing sticker is removed.
- Please make sure the battery is inserted correctly.

Accessory Kits

- Ear tips: 3 pieces
- Battery: 2 pieces
- Ejector tool: 1 piece
- Cleaning tool: 1 piece

02

Unit description: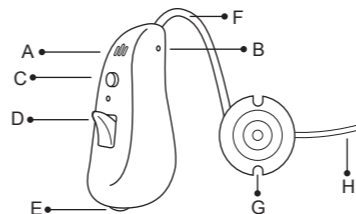


- A. Microphone
- B. Pin hole (to eject ear tube)
- C. Mode Button
- D. Volume adjuster
- E. Battery Compartment
- F. Ear tube
- G. Ear tip
- H. Concha lock


03

Initial use and operation

1) Turn on: Please insert the battery ("+" symbol upright) in battery compartment and close the battery compartment. After a few seconds, the device will automatically be in minimum volume and in standard mode.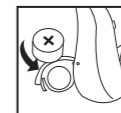

Turn off: Please open the battery compartment.

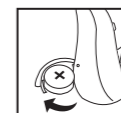

2) Please refer to the below indicator for fitting into your right or left ear.

04


- marker "2B" in red = right ear
- marker "2B" in blue = left ear

3) Please choose a suitable size ear tip, and then place the earphone (with an ear tip connected) into the ear.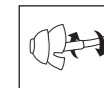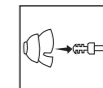

4) Place the main body of the unit behind the ear, so the device hooks over and rests behind the ear. Twist it slightly until fitted comfortably, and then please place the concha lock inside your ear carefully, to help fix the earphone in your ear correctly.

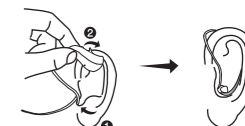
05

5) Gradually increase the volume by the volume adjuster until the sound is at a comfortable level. If there is an echo, it may indicate the volume is too high.

6) There are 4 different modes: Standard, Noise reduction, Indoor and Outdoor; you can adjust it by pressing the Mode button, and a sound (i.e. "di") will indicate which mode it is:

- Standard mode: "di" (once)
- Noise reduction mode: "didi" (twice)
- Indoor mode: "didi" (thrice)
- Outdoor mode: "didi" (four times)

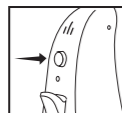
06

7) How to change ear tube

1. Use the ejector tool to push out the pin (which is used to fix the ear tube with main body).

2. Remove the right ear tube (if current one is right), and switch to the left ear tube instead.

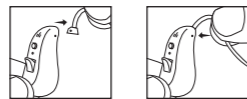3. After, put the pin inside the hole slowly, and then use the opposite of ejector tool to push the pin inside until it is firmly in place.

07

Maintenance and cleaning

Cleaning

To ensure optimal use, regularly clean your hearing amplifier, ear tube & ear tips as necessary.


1) How to clean the hearing amplifier device:

Please use a soft & dry tissue or cloth to clean it. DO NOT wash it with water (with the battery compartment open and/or detached ear tube).

2) How to clean the tube:

Please use the provided cleaning tools to clean it. If very dirty, please use a soft & dry tissue or cloth to clean it.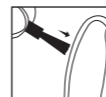

08

3) How to clean the ear tips:

Please use the provided brush to clean the ear tips slightly

Storage

During long periods of non-use, store your hearing amplifier with the battery compartment open (and batteries removed) in a drying system; in order to prevent the penetration of moisture.

Disposal

Please dispose of the device in accordance with EC Directive-WEEE (Waste Electrical and Electronic Equipment). If you have any queries, please refer to the local regulations.

09

Troubleshooting

Problem	Solution
No sound or weak sound	- Turn on the device; - Remove and re-install the battery; - Replace the battery; - Clean or replace the tube or earplug; - Contact your distributor;
Hearing Amplifier emitting an alert signal	- Replace the battery.
Sound is distorted	- Replace the battery; - Decrease the volume level; - Clean or replace the tube or earplug.
Hearing Amplifier does not work	- Turn on the device; - Make sure the battery compartment is closed well; - Replace the battery.

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