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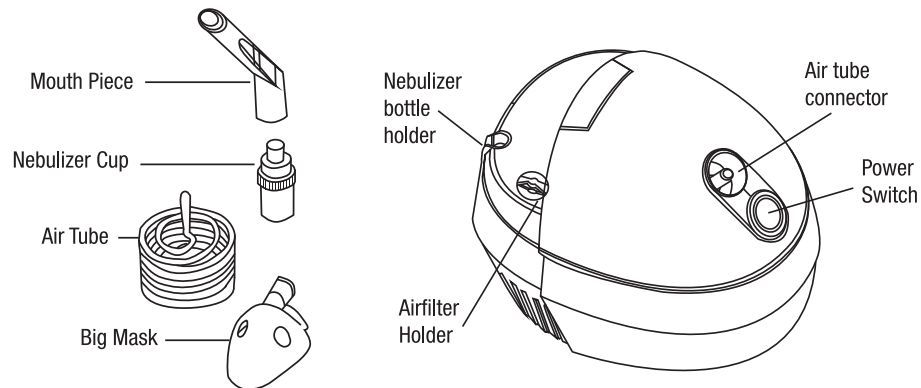
EU authorised  
representative:  
Shanghai International  
Holding Corp. GmbH (Europe)  
Add: Eifflstrasse 80,  
20537 Hamburg, Germany

 Huizhou Jinghai Medical Technology Co., Ltd.  
Floor 6, Huicheng Industry Building, No.9. Huifeng Dong'er Road,  
Zhongkai High-techZone, Huizhou City, Guangdong Province, China

## EN I Compressor Nebulizer

PR-821

### 1. What is the Nebulizer?

The Medical Air Compressor Nebulizer System is a medical device designed to deliver medication prescribed by a physician to the airways of the lungs. The nebulizer and air compressor combined provides an aerosol mist for inhalation by the patient for medical treatment only.

**2. How Does the Nebulizer Work?** This unit is compact, simple to operate and convenient to carry. It uses a non-oil lubricating, single-cylinder piston pump to create air pressure. Air flows through the air filter into the Nebulizer's air compressor unit, out the jet outlet, and then through the Nebulizer. The main air flow into and out of the air compressor unit.

**3. Contents in the box:** 1x Nebulizer compressor 1x Air tube 1x Mouth piece  
1x Adult mask 1x Children mask 1x Medication cup 5 x Air filters

4. Components The Standard Nebulizer components are diagramed below in Figure

**5. How Do I Insert Medication into the Nebulizer?** To insert medication into the Nebulizer, do the following: 1. Open the cap (see Figure). 2. Fill the cup with medication as prescribed by your physician. Be sure that the medication does not exceed the liquid site line of 6 ml. 3. Close the cap. 4. Connect the mouthpiece/Mask.

**Note:** The mouthpiece is to be used for oral inhalation only.

**6. How Do I Operate the Nebulizer?** To operate the Nebulizer, do the following:

1. Plug in the air compressor, but do not connect the Nebulizer. 2. Make sure that the air filter is attached to the air compressor. 3. Turn on the power switch. 4. After the air compressor runs properly for 1-2 minutes, connect the jet outlet of the air compressor to the inlet of the Nebulizer by the air tube. 5. Begin treatment according to your physician's instructions. 6. Inhale and exhale naturally. Be sure to stop when the medication is completed. 7. When you have completed use, turn the air compressor unit off before you disconnect the air hose.

**WARNING:** Do not use near flammable objects or locations near explosive materials.

### 7. What Do I Do After I've Used the Nebulizer?

To keep your Nebulizer working properly, after each use do the following: 1. After you have disconnected the air hose, disconnect the mouthpiece/mask 2. Open the cap and empty the cup. Wash the Nebulizer with running water or soak in warm water for 15 minutes. For a more thorough, hygienic cleaning, add some vinegar in the water. 3. Dry the Nebulizer completely before storing it. NOTE: Do not boil the nebulizer. Doing so could damage it.

### 8. When should I Replace the Air Filter?

Generally, the air filter should be replaced every three months or after 300 uses. To replace the air filter: 1. Remove the filter cover. 2. Replace the filter. 3. Re-attach the filter cover. Note: Whenever the filter becomes wet, allow it to dry completely before using.

### 9. What Maintenance for Storage Does the Nebulizer Require?

1. Always handle your Nebulizer with care.
2. Disconnect the power plug from the wall outlet when not in use.
3. Keep your Nebulizer out of children's reach.
4. If you store your Nebulizer, do not bend the air hose.
5. If you store your Nebulizer, try to keep it in the following environmental ranges:
  - 1) Temperature: -40°C - 55°C.
  - 2) Relative humidity: 95%.
  - 3) Atmospheric pressure: 0.05MPa 0.106MPa.
6. If possible, store your Nebulizer in a well ventilated room.

### 10. Specifications:

Electric requirements: AC 220V/50Hz	Working compressor pressure: 35~40 PSI
Compressor free air flow: >10L/min	Sound level: <60 dB(A)
Nebulizer net weight: 1.5kg	Compressor size: 288x180x100mm
Electric safety: UL547 Class B	Average Nebulization Rate: 0.4mL/min
Ambient temperature range: 5°C/40°C	Medication bottle capacity: 7mL

### 11. Troubleshooting guide. Symptom/Probable Cause /Solution

Too much noise: No filter in the unit: Put filter in the unit

The nebulizer does not run: • Mai compressor switch is off: Turn the unit on.

- Nebulizer was not cleaned after its previous use: Clean the unit
- Air hose is bent. Remove any bends or kinks in the air hose,
- The filter clogged. Replace the filter.
- No medication is left. Add the appropriate amount of medication to the cup.
- The cap is not attached properly. Re-attach the cap.

Water Drops have formed on the air hose. The medication was over-filled:

Add appropriate amount of medication to cup, connect the air compressor, and turn the unit on.

- The Nebulizer was washed but not dried. Open and close the Nebulizer repeatedly the drops disappear. **NOTE:** Please contact your dealer for assistance with any other difficulties.

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