TPF Family[®]

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Safety and usage instructions for the individual items

Nasal aspirator: **Warning:** For adult use only. Choking hazard due to small parts. The tip or other removable parts of the nasal aspirator may pose a choking hazard. Check that all parts are securely attached and the device is intact before each use. **Safety instructions:** Do not insert the tip too deeply or with pressure into the nostril to avoid injury to the delicate nasal mucosa. Do not use the nasal aspirator if there is a nosebleed or visible injury in the nose. **Usage Instructions:** Gently place the soft nasal tip at the entrance of one of your child's nostrils, ensuring a seal. Squeeze the balloon completely until no air escapes. Slowly release the bulb to create gentle suction that draws mucus into the bulb. Soothe your child during and after use.

Nail scissors: **Warning:** Risk of injury due to sharp blades! For adult use only. **Safety instructions:** Use the scissors only when your baby is calm and relaxed. Ensure good lighting. Hold your baby's hand or foot securely while cutting. Do not cut too close to the skin to avoid injuries. The rounded tips are for added safety but do not replace caution. **Usage instructions:** It's best to trim nails when your baby is sleeping or very calm. Gently press the fingertip away from the nail to protect the skin. Trim fingernails following their natural curve. Toenails should be cut straight to avoid ingrown nails.

Nail clipper: **Warning:** Risk of injury due to sharp blades! For adult use only. **Safety instructions:** Same as for nail scissors (see above: calm baby, good lighting, secure hold, do not cut too close to the skin). **Usage instructions:** Same as for nail scissors (see above: timing, press fingertip away, cutting shape). Position the nail between the clipper's cutting edges.

Comb and brush: **Safety instructions:** Check that no bristles or teeth are loose before use to prevent swallowing. Avoid applying pressure on the baby's sensitive scalp. **Usage instructions:** Gently comb or brush your baby's hair. For tangled hair, hold the strand at the root to avoid pulling. Ideal after bathing when hair is still slightly damp, or for daily care.

Glass nail file: **Warning: Fragile!** For adult use only and with great caution. **Safety instructions:** Risk of breakage. Glass nail files can break easily if dropped or mishandled. Broken files have sharp edges and shards that can cause serious cuts or, if swallowed, internal injuries or choking. Check the glass file for cracks, chips, or damage before each use. **Usage instructions:** Use only when the child is calm and lighting is good. Hold your baby's hand securely. File with very gentle pressure and always in one direction. The glass file is often more effective than standard files, requiring less pressure and fewer strokes.

Nail file: **Warning:** For adult use only. **Safety instructions:** Use only gentle pressure to avoid injuring the sensitive skin around the nail. Do not file off too much of the nail. **Usage instructions:** Hold your baby's hand securely. Gently file in one direction to smooth the nail and remove sharp edges.

Pipette: **Warning:** For adult use only. **Safety instructions:** Choking hazard: The pipette tip or suction bulb may come loose and pose a choking hazard. Check that all parts are securely attached before each use. Never leave your child unattended with the pipette. Do not insert the pipette tip too deeply into the mouth or nose. **Usage instructions:** Squeeze the suction bulb, dip the tip into the liquid, and slowly release to draw up the liquid. Carefully insert the tip into the corner of the child's mouth and slowly squeeze the bulb to release the liquid drop by drop. Allow time for the child to swallow.

Measuring spoon: **Warning:** For adult use only. **Safety instructions:** Check the temperature of the food/liquid before each use to avoid burns. Never leave your child unattended with the spoon. Ensure the spoon has no sharp edges or damage. **Choking hazard:** Make sure no parts can break off. If the spoon is very small, there is a risk of the entire spoon being swallowed. **Usage instructions:** Fill the spoon only with the intended amount. Gently guide the spoon to the child's mouth.

Finger toothbrush: **Warning:** For adult use only. **Safety instructions:** Contains small parts. Choking hazard! **Usage instructions:** Place the brush like a fingertip on your index finger. Gently rub it over the gums and the first small teeth – ideal for soothing during teething and for early dental care.

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TPFFamily – eine Marke der Traumpreisfabrik Hornung GmbH
Traumpreisfabrik Hornung GmbH