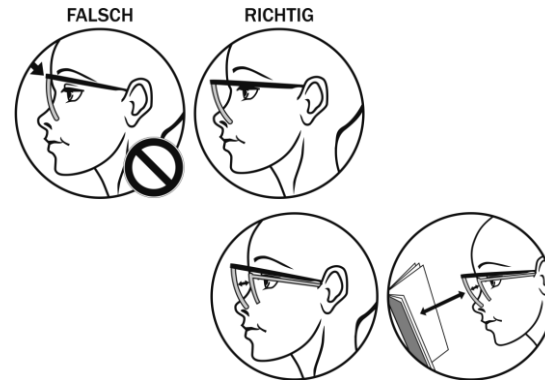
DEUTSCH

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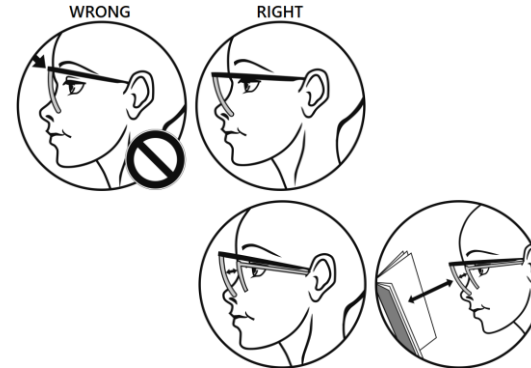
ENGLISCH

SAFETY ADVICE

- not suitable for use in traffic
- not suitable for direct look into the sun
- no protection for artificial light sources
- not suitable for use as safety glasses
- interrupt use if signs of discomfort occur
- not suitable as replacement for glasses with corrective lenses
- no toy
- not suitable for long distances
- only suitable for reading and short distances

MAINTENANCE & CLEANING

- Do not clean using solvents or alcohol
- To remove dirt or fingerprints use the included cleaning bag or a soft cloth
- To remove persistent dirt from the glasses use water, a neutral or soft detergent
- Applying too much force during cleaning may cause scratches

WEAR OVER GLASSES

Magnifying glasses can be used separately or over glasses with corrective lenses or contact lenses. The magnifying glasses should be worn on the tip of the nose and **NOT** in the same position as normal glasses.

USAGE OVER GLASSES

To find your best focus leave some space between the magnifying glasses and your sunglasses or glasses with corrective lenses.

FINDING FOCUS

Hold an object in 30 to 40 cm (approx. 12 to 16 inches) distance to the magnifying glasses. Vary the difference until you can see the object in focus.