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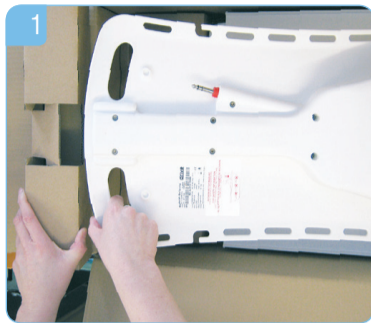
ERSTINSTALLATION:

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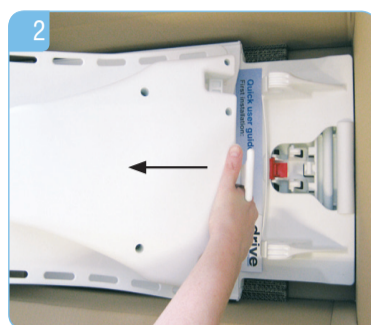
Quick User guide

FIRST INSTALLATION:

1. Remove from the carton and check stop lever


1. Raise the drive unit

2. Move the backrest forward

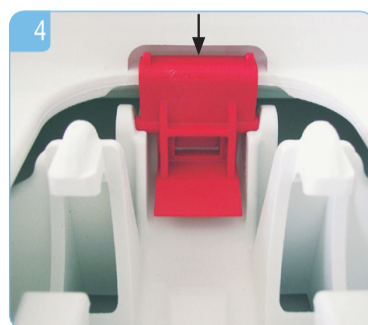3. Raise the seat unit

4. Please ensure that the red seat unit locking mechanism is engaged by pushing down on the seat.


2. Attaching the backrest into the seat unit


1a correct

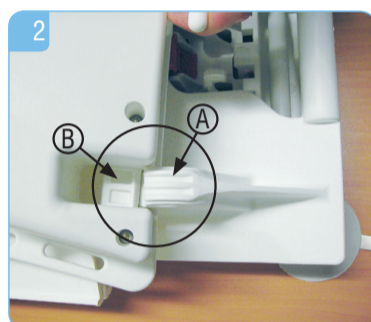
1b wrong

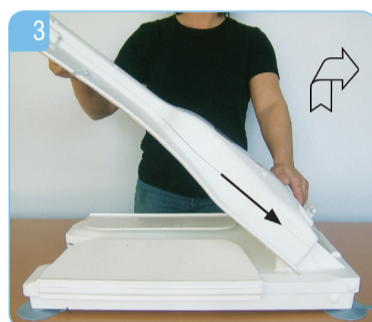
2. **A: Bearing block – B: Guide pin**
Slide the backrest into the hinge located on the back of the seat unit.

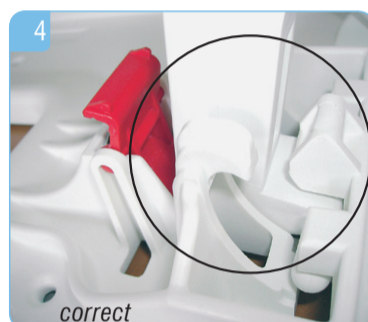3. Lift the backrest to an 45° Angle until it clicks into the latch.

correct

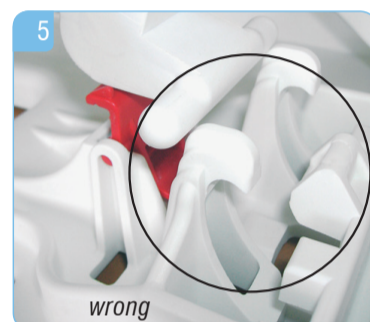
wrong


1. Important:
Please ensure that the actuator is fully retracted in the backrest.

4. Important:
Please ensure that the actuator end is/sits properly in the base plate's ramp.

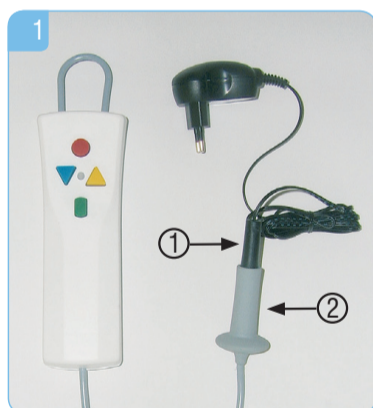3. Charging the battery • Attaching the hand control

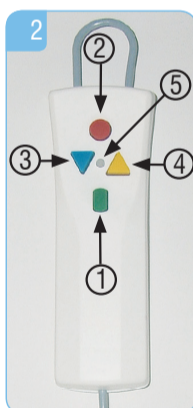1. Place the chinch 1) of the charger into the socket of the cable 2) attached to the hand control. During the charging process, the red light will be on. The light changes to green when the battery is fully charged.


2. Hand control description:


1. Green: power On button
2. Red: emergency Stop
3. Blue: down button
4. Yellow: up button
5. Control light

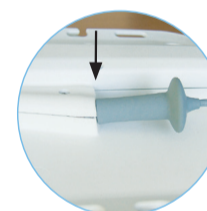
3. Plug the hand control to the bathlift (at the back of the backrest). Make sure that the cable is properly seated on the jack. The red ring at the jack is now hidden.

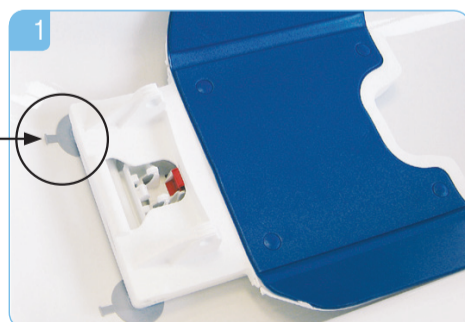4. Placing or removing the bathlift into/from the bathtub

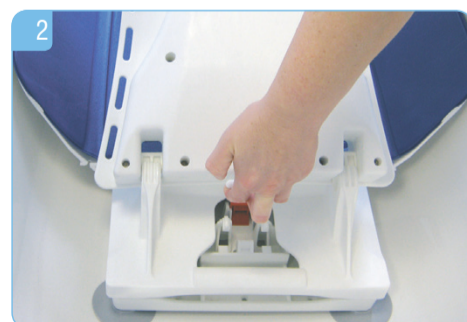1.
 - Remove anti-slip mats or similar utilities.
 - The tongues of the rear suction caps must be directed outwards.
 - The suction cups at the bottom of the Bellavita must be clean and dry.
 - Make sure that the rear suction cups are located close to the bevelled edge of the bath.

2.
 - Attention! It is very important that both guide pins are inserted into the bearing blocks! (see point 2).
 - Pivot the attached backrest up until it reaches its final position.
 - **The backrest must not collide with the bathtub. In that case you have to reposition the bathlift.**

3.
 - Before usage, bring the lift into a position which allows seating comfortably.

4.
 - In need of elimination please turn 45° In advance and pull out. Remove backrest according point 2.
 - Remove the two rear suction cups by pulling the tongues. The seat unit can now be removed from the bathtub.