## D DUSCHHOCKER 33035

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## GB BATH SEAT

### 33035

#### DIRECTIONS FOR USE AND SAFETY INSTRUCTIONS

##### General notes

- Read the instructions carefully before use.
- Keep these instructions for future reference and include these instructions if you pass on the product to third parties.
- Use the product only for its intended purpose. Any other use as well as modifications of the product have to be considered as improper use and could cause injuries or product damages. The manufacturer will not be held responsible for any damages resulting from such improper use.
- Not for commercial use.

##### Safety notes

##### WARNING

Life and accident risk for toddlers and kids! To avoid danger of suffocation, keep packing materials away from babies and children. Risk of suffocation due to packing material. Small parts can be swallowed! Children often underestimate the dangers!

- Max. load capacity 150 kg.
- Don't used as a step.
- Make sure that the screws are fix tight on the seat.
- The bath seat must stand straight and securely.

##### ATTENTION

Risk of injury from sliding down. Please move carefully while using.

##### NOTE

The use of soap, shower gel, shampoo etc. increases the risk of slipping!

##### Assembly instruction

1. Lay down the seat (1x) with the back side in front of you.
2. Take the two legs (2x) and place them crosswise on top of each other (rubber feets upwards, flat area on top of each other). Then fix both legs with the screws and washers (each 5x) in the middle of the seat.
3. Check that all screws are tight.
4. Disassembly in reversed order.

##### Adjusting the seat height

- Push the locking buttons on the legs and choose the desired height. The seat height can be adjusted 8 times.
- Make sure that the locking knobs snap again!
- Always choose the same height position for all 4 legs.

##### Cleaning and care

To avoid scratches on the surface wipe the components with soap water and a soft sponge, then wipe dry with a damp cloth.

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Haushalt International Im- & Export GmbH