# HEALTHSON

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**Lumbara® Dekompressions-Gürtel**

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## USER MANUAL

### Lumbara® Decompression-Belt

Thanks for choosing our Lumbara® back belt.  
Please read this manual carefully before using

Say goodbye to your back pain!

Designed in Germany

## One Size For 29 Inches To 49 Inches Waists

BELT LENGTH	WIDTH BEFORE INFLATION	WIDTH AFTER INFLATION
45.2 inch	5.1 inch	7.9 inch

### Specification

**Gross weight:** About 1.71lb (Package including)

**Inner layer of air inflation:** TPU (Special soft)

**Outer layer:** PU

**Pumping times for full inflation:** Usually 30-45 times per usage

**Pressure limits:** 0.19MPA

### Product Details

### Hand Pump


To inflate the belt

Clip the hand pump to the air valve to inflate the belt.

## Air Valve Key

It can be inserted into the hose connector to release the air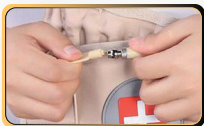

A pocket on the front of the belt holds a plastic "Valve Key" that fits into the hose on the deflation. The key is further secured with elastic, a very thoughtful feature allowing you to deflate the belt at a moment's notice, and to keep the hose in top condition by not having to wrangle up a substitution that doesn't fit as well - this key will not get lost.

## Extension Pad

To make the belt length fit a larger waist

Take out the Extension Pad and stick it on our belt to make it for a larger waist

## Wearing time

- Start with 15-30 minutes per session and gradually increase the time.
- Do not use the belt during sleep or for long periods of time without rest.

## Tips for optimal results

- Use the belt regularly to achieve long-term benefits.
- Combine use with light stretching exercises for maximum effectiveness.
- Explore our additional products for holistic back health.

## Our recommendation

Posture correction belt and decompression neck brace

### 1. BackHelper® posture corrector belt:

- Improve your posture and reduce tension.
- Wear the belt discreetly under clothing for a long-lasting effect.

### 2. NeckRelax® decompression neck brace:

- Relieve tension in the neck area and promote relaxation.
- Simply put on and inflate for targeted decompression.

## Special offer for you

Purchase our posture correction belt and decompression neck brace together now and receive an exclusive **20% discount!** Enter the code "**BACK20**" at checkout.

We hope you experience a noticeable improvement in your back health with your decompression belt and additional products. If you have any questions or concerns, please do not hesitate to contact our customer service team.

Thank you for your trust in our products!

Your back health is important to us.

## FAQ

PROBLEM	SOLUTION
Difficult to fill in the air	Check the position of the clip and air valve is correct
Can't fill in air	Check if the air valve is tightened
After inflation, the pump has a sound of air leakage	It means the belt has been inflated fully, air pump limits the air fill in
After wearing the belt couldn't be inflated fully	User wear the belt too tight, loosen it a bit, it will be in good condition
How often to wear the belt	We recommend wearing them for 15 to 60 minutes a day & during strenuous activities.

## Several things to consider

1. This belt cannot be immersed in water to wash it.
2. This belt should not be exposed to direct sunlight and heat, please store it in a cool place after use.
3. When the belt is inflated completely, don't inflate it again or the belt might get damaged.

## **Please don't use this product under following conditions**

1. Fracture: Thoracic vertebra, lumbar, sacral vertebrae, pelvis, rib.
2. Serious rheumatoid arthritis, rupture of ligament.
3. Purulent tuberculous spondylitis.
4. Lumbar disease caused by tuberculosis or tumor.
5. Ganglion or skeleton lumbar.
6. Serious cardiovascular disease and respiratory disease.
7. Pregnant woman, children.
8. Serious osteoporosis patient.
9. Serious skin disease.
10. The patient who has acute stage lumbar trauma.
11. Please follow up the doctor's advice if you have other disease.

## **We offer 100% SATISFACTION GUARANTEE**

Your satisfaction is our priority. Enjoy a worry-free shopping experience with our 100% satisfaction guarantee and 14-day return policy. If you have any questions, our customer support team is always available by email:

 [support@healthson.de](mailto:support@healthson.de)

# HEALTHSON