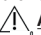
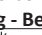
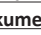
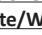
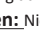
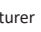
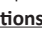
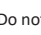
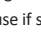
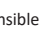
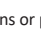
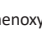
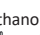
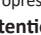
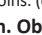
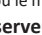
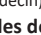
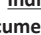
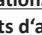
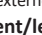
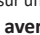
www.axion.shop/symbols

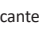
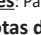
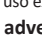
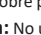
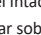
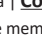
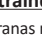
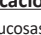
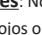
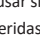
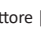
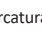
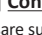
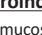
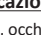
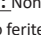
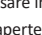
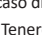
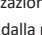
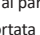
Anwendung | Application | Application | Aplicación | Applicazione | Toepassing |  
Приложени | Primjena | Εφαρμογή | Aplikace | Ansøgning | Taotlus | Hakemus | Ansökan |  
Ansökän | Aplicație | Pieteikums | Aplikasjon | Aplikacja | Aplicação | Aplikácia |  
Aplikacija

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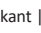
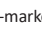
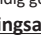
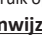
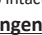
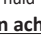
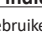
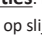
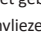
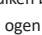
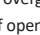
**GBR | INSTRUCTION FOR USE ELECTRODE GEL:**  Conformity marking |  Manufacturer |  UDI marking |  Observe instructions for use | **Intended use:** Electrode-accessories  
(Contactgel) for optimal transmission of electrical impulses (conductivity) in connection with TENS and EMS electrodes for therapeutic and diagnostic medicine. The product is intended for OTC use. The pa-  
tient group is not restricted, if necessary, by the user (doctor) | **Indications:** For external use on intact skin | **Contraindications:** Do not use if sensible to parabens or phenoxyethanol |  **Attention**  
**- Observe accompanying documents/warning notes:** Do not use on mucous membranes, eyes or open wounds | Keep away from children | Do not clean nor disinfect |   Store in a  
cool and dry place, do not expose to direct sunlight |  Minimum shelf life |  Only use within the shelf life after opening. |  Non-sterile | **Ingredients:** Water, carbomer, sodium hydroxide, sodium  
chloride, propylene glycol, phenoxyethanol, methylparaben, ethylparaben |  Does not contain formaldehyde |  medical device |  Batch code |  Article number

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