Sitzkissen zum Druckmanagement Seat cushion for pressure management

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WITH COCCYX CUTOUT

Intended Use:

- The feela.® pressure management seat cushion relieves pressure and increases sitting comfort.
- It is suitable for use in office chairs and other flat seating surfaces - or even when traveling.
- Due to the improved pressure distribution, it is also used for the prevention and aftercare of decubitus.
- The U-shaped cut-out provides pressure relief for the coccyx.

Intended user group: Persons of adult age

Indications:

- Tailbone pain
- Threatening or healed decubitus ulcer
- Promotes blood circulation

Instructions for use:

- Place the cushion on a flat surface with the anti-rusch pimples facing down and the coccyx recess facing back.
- After 30 seconds of use, the feela.® cushion will adapt to your body shape using the warmth of your body - ideally at room temperature.
- Continuous use should be interrupted by a short break after three hours.

Safety Instructions:

- Keep seat cushions away from fire-inducing heat, sparks or open fire.
- Only place the seat cushion on insensitive surfaces - not suitable for bright leather (without intermediate layer).

Contraindications:

- Not to be used for unstable fractures in the spinal column and pelvic area.
- Open wounds
- Lymphatic foot disorders, sensory and critical circulatory disorders of the supplied body region, PAVK
- Disease patterns in which the attending physician or therapist advises against their use

Specification of technical data:

- Dimensions (width x height x depth): 45 x 7 x 36 cm
- Weight: 0,7 kg
- Density: 50 kg/m³
- Recommended user weight:
 - Soft: 50-80 kg
 - Medium: 60-100 kg
 - Hard: 80-130 kg

Information on cleaning:

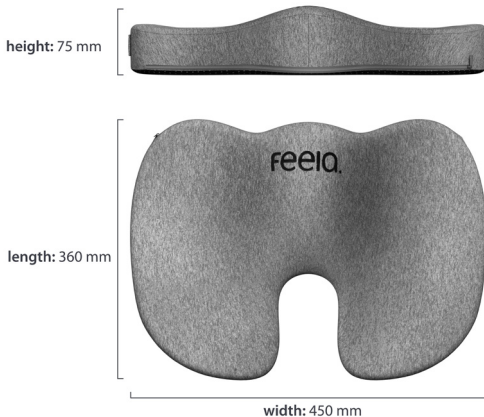
- Open the front zipper and remove the foam core. The inner cover is not washed and remains on the core.
- Wash the cover at 40 °C in the washing machine.

Intended service life:

- 5 years

Storage and transport conditions:

- Constant conditions, ideally at temperatures of 18 to 24 °C and relative humidity of 40 to 60%.

Instructions for safe disposal:

- Not recyclable, disposal as plastic waste

Information to authorities in case of "events":

- All serious incidents occurring in connection with the seat cushion must be reported to the manufacturer and the competent authority:

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Further information on the application:

- In case of problems with the application, please ask your specialist dealer or the responsible feela. customer advisor for advice. Phone: +49 991 9992267

Information on repeat use:

- The feela.® seat cushion is not intended for reuse.

Scope of delivery:

- 1 x feela. seat cushion for pressure management
- 1 x cushion cover
- Also: instructions for use, protective packaging

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Typesetting and printing errors expected.
Changes in colour, material and construction possible.