INSTRUCTION SHEET - 0.2 L MINI HOT WATER BOTTLE

(GB)

Dear customers,
Congratulations, you have purchased a German quality product. We have compiled some important information for you. Please read this carefully to ensure that you will be able to enjoy this product for a long time. Retain this instruction sheet should you require any information contained on it in the future. Should you pass the item on to another person, please ensure you also pass on the instruction sheet.
Hugo Frosch GmbH

Instructions for use

Hugo Frosch hot water bottles are suitable for relaxing, against cold feet and to increase your overall well-being. General warning! Particular care must be taken when the hot-water bottle is used by persons who are unable to heat, persons who have arthritis as well as circulatory problems.

Certain diseases such as diabetes may cause temperature sensitivity disorders. Disabled or infirm persons may not be able to draw attention to the fact that they are too warm. When using the hot water bottle with infants, you must ensure that the temperature of the water is no higher than 40°C. Always check that it does not result in overheating. As babies cannot change their position independently, they are completely reliant on the attentiveness of their parents. If you have any doubts, please contact your doctor before using the hot water bottle.

Using

When filling this hot water bottle, do not use boiling water and fill to a maximum of two-third capacity or less. Do not overfill as this might cause the bottle to burst. Hold the bottle by the neck in an upright position and fill slowly to avoid hot water splashing back. Squeeze the bottle carefully to expel air, until water appears at the opening. Tighten sufficiently to ensure that no water escapes. Finally, make sure the funnel is empty. If there

Check the bottle and stopper regularly for signs of wear and damage. If you notice any damage, or after a maximum of 5 years, replace it with a new one.

Manufactured from high-quality thermoplastic – free of phthalate – tested for harmful substances – recyclable – odourless – safety tested and monitored.

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Vaša tvrtka Hugo Froesch GmbH