- Ⓛ Montageanleitung
- Ⓜ Assembly Instructions
- Ⓝ Instructions de montage
- Ⓔ Instrucciones de montaje
- Ⓜ Istruzioni per il montaggio
- Ⓝ Montage-instructie

- Ⓟ Instruções de montagem
- Ⓞ Montagevejledning
- Ⓢ Monteringsanvisning
- Ⓟ Instrukcja montażu
- Ⓜ Οδηγία μοντάζ
- Ⓒ Návod k montáži

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Ⓒ Bathroom stool with armrest and backrest, Secura Premium

Model: 23546\*

**Assembly Instructions:**

1. Fasten the two holders for the backrest F and G and the holders of the armrests E using the screws H into the threads of the seat provided C. Make sure that the screws are firmly seated.
2. Push the adjustable legs D into the openings provided in the seat by pressing the button. The leg is correctly engaged when the adjustment button comes out in the hole provided with an audible click.
3. Turn the stool over and push the armrests B into the holder E and screw the armrests B into the holder E using the screws I. Make sure that the screws are firmly seated.
4. Push the backrest A into the holders F and G and fasten the backrest A using the screws I into the two holders F and G. Make sure that the screws are firmly seated.

**CAUTION!**

Including mounting material. The enclosed mounting elements must be used as described in the installation instructions. The seat is height adjustable by means of the buttons on the legs. Please make sure that all legs have the same height setting before using the seat. Before subjecting the seat to loads, please ensure that all screws have been tightened and that all locking buttons are engaged.

max. load capacity 130 kg.  
Recommended service life: 15 years from production date.

**Areas of use:**

This product may only be used in private households! Not suitable for use in rehabilitation facilities and for the medical care of persons. This product is intended for use in the wet area of the shower.

**Care instructions:**

Clean with a damp cloth.  
Do not use any sandy abrasive agents or scrub sponges for cleaning.

**Safety instructions:**

**Pay attention to a firm hold! Smooth or wet surfaces can worsen the non-slip properties. Check connecting elements regularly. The product should be checked regularly for stability and damage, and reuse is only permitted after any necessary repairs have been professionally carried out. Please keep these instructions, as they contain important information. Do not step on the seat.**

**Do not use to lift persons!**  
Accidents resulting in serious injuries may occur if the handles are used for incorrect purposes e.g. as gymnastic equipment or steps. Wenko Wenselaar GmbH & Co KG shall not assume any liability whatsoever for incorrect use!

**Exclusions:**

Wenko does not assume any liability for damage to persons and property caused by improper mounting, incorrect use of the handles, in particular by non-observance of the warning and safety instructions listed above.

**For after-sales service please contact the manufacturer directly:**

Wenko-Wenselaar GmbH & Co. KG  
Im Hülsenfeld 10 · 40721 Hilden, Germany  
Tel: +49 2103 573-0 · Fax: +49 2103 573-190  
service@wenko.de · www.wenko.de

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