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Nasivent Tube Plus - Simply breathe better!

Nasivent Tube is a high-quality silicone nasal dilator to improve nasal breathing and prevent snoring. With proper use and care, you will enjoy this product for a long time.

Nasivent Tube Plus is inserted into the nose before sleeping. It gently stretches the outer nasal walls and thus facilitates nasal breathing. Improved nasal breathing reduces the tendency to breathe through the mouth and therefore the risk of snoring. A good oxygen supply contributes to a more peaceful and restful sleep.

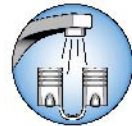
Nasivent offers a pleasant wearing sensation. Due to its flexibility and softness, as well as the 5 different sizes, Nasivent adapts excellently to the different shapes of the nose. The cones on the top of the tubes improve the hold in the nose.

Areas of use:

For use as a remedy for snoring; to improve nasal breathing; to optimise oxygen supply during sports; for blocked nose; as an alternative to nasal spray

Material: High-quality silicone; BPA-free

Safety note: Do not cut the connecting bar of the tubes! Not suitable for children under 6 years of age.

Clean the Nasivent Tubes with soap and water before use.

Insert the Nasivent Tubes completely into the nose, making sure that the retaining pins point upwards.

If the Nasivent tubes are too long, you can simply shorten them with scissors.

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