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133 mm

81 mm

Video-Anleitung

 Scan me

ANLEITUNG:

1. Zähne putzen

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INSTRUCTION:

1. Brush your teeth

2. Apply the whitening gel with the Whitening pen (about 2-3 mm thick)
Important: Please 15 seconds turn at first dawn

3. Plug in your smartphone and put the LED piece in your mouth

4. Apply for 15 minutes

5. Remove whitening gel & rinse your mouth

6. Cleaning the LED-piece

TIPS:

1. Brush your teeth with the Unique-Smile Activated Powder. Perfect preparation for the bleaching. Removes coarse impurities.

2. Dry your teeth before use the gel

3. Apply in the evening before going to bed

4. During the bleaching application, don't drink tea, coffee, wine and don't smoke

5. Use Mouthwash -> removes almost all bacteria from your mouth

6. Recommendation: one week - use 3 times in a day to get an optimal result.
Note: Applies in particular to smokers and coffee drinkers.

Warning:

The Gel does not lighten crownes, teeth filling or dentures. Do not use on children under 12 years of age. Keep out of reach of children. Do not use during pregnancy and breastfeeding, gum disease, periodontal disease, bleeding gums, gingivitis, receding gums and sensitive teeth. Do not swallow. If gum irritation occurs, stop the procedure and consult with your dentist.

We recommend continuous use for no longer than 4 weeks. Than take an equally long break. Avoid direct light into the eyes.

Content:

3x 2ml Whitening pen, 1x LED Piece, 4 adapter: iphone, USB, USB-C, micro USB, 1x shadeguide, 1x manual

IT/FR/ES

Scan me to change Language

100%
VEGAN