# Insect Bite Healer Instruction Manual

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# INSECT BITE HEALER INSTRUCTION MANUAL

Introduction .....	2
Foreword .....	2
Package Contents.....	2
Product Description .....	3
Contraindication .....	4
General Warnings .....	4
Power Connection .....	7
Usage .....	8
Safety Notes .....	10
Explanation of Symbols .....	11
General Precautions .....	13
What If There Are Problems? .....	14
Cleaning and Care .....	15
Technical Specifications .....	16
Warranty and Repair Terms .....	17
EMC Information.....	18
Electromagnetic Emissions.....	19
Electromagnetic Immunity.....	20

## **Introduction**

Thank you for purchasing this Insect bite healer.  
Please read the Manual carefully before using the device, and put it in a safe and secure place for reference.

## **Foreword**

The Insect bite healer can be used to relief itching and swelling caused by insect bites or stings, which is suitable for over 3 years of age under supervision, can be used for self-treatment by anyone over 12 years of age.

## **Package Contents**

<b>No.</b>	<b>Name</b>	<b>Quantity</b>
1	Insect bite healer	1
2	Manual	1

# **Product Description**

The Insect bite healer is a light weight, portable, hand held, battery powered, user-operated device that produces mild heat for direct contact with the affected areas of the skin.

## **Intended Use**

The Insect bite healer is intended for relieving itching and swelling caused by insect bites or stings by applying heat (local hyperthermia).

## **Intended User**

Healthcare Professional & Lay person.

## **Patient target group**

The Insect bite healer is suitable for people who want to relieve the itching and swelling caused by insect bites or stings. Over 3 years of age under supervision, can be used for self-treatment by anyone over 12 years of age.

## **Intended Use Environment**

Hospital/Healthcare Facility & Home.

## **Working principle**

The principle of Insect bite healer is to use heat energy to act on the surface of the skin, so as to reduce or eliminate the itching and discomfort caused by insect bites. When the surface of the insect bite site is briefly concentrated heat action, the local high heat will stimulate the skin nerve endings, so that they react to the tissue toxins in the insect saliva. At the same time, the heat effect can also promote local blood circulation, accelerate the metabolism and excretion of toxins, reduce itching and discomfort.

## **Contraindication**

- Do not use the device if you suffer from diabetes. Sensitivity to pain could, in certain circumstances, be reduced and diabetics may burn themselves.
- Do not use the device on people with disabilities, children under 3 years old or people with reduced sensitivity to heat. This includes people with skin alterations due to illness or scarred tissue in the application area, after taking pain relief medication, drinking alcohol or taking drugs.
- Do not use the device on acutely or chronically diseased (injured or irritated) skin (e.g. inflamed skin, whether painful or not, reddened skin, rashes, (e.g. allergies), burns, frostbite, bruises, swellings (without signs of an insect bite/sting), both open and healing wounds, and surgical scars where the healing process could be affected).
- Do not use the device if you are suffering from a fever.
- Do not use the device on sensitive areas of skin.
- Do not use the device if you have any sensory impairment that reduces the feeling of pain (e.g. metabolic disorders).
- Do not use the device if you are using lotions, creams and gels locally at the same time.
- Do not use the device if you suffer from persistently irritated skin due to long-term heat application on the same area of skin.

## **General Warnings**

- Over 3 years of age under supervision, can be used for self-treatment by anyone over 12 years of age.

- Before using the device, consult your doctor if you have any skin conditions, serious illness (in particular if you have a propensity to thromboembolic conditions or recurrent malignant growths), unexplained chronic pain in any part of the body.

- The device is not a substitute for medical consultation and treatment. If the symptoms persist despite using the Insect bite healer and stop using it. Seek medical advice immediately if the symptoms worsen whilst or after using the Insect bite healer and stop using it.

- Seek medical advice immediately if it is a tick bite.

The bite may result in transmission of pathogens such as tick-borne encephalitis (TBE) or borreliosis.

- Seek medical advice immediately if the first signs of increased itching, skin redness, swelling, dizziness, nausea, shortness of breath, formation of wheals or a drop or increase in blood pressure occur.

- Do not use near to the eyes, on the eyelids, mouth or on mucous membranes. Furthermore, the device must not be used on moist skin.

- Do not allow children to play with the device.


- Store the device out of the reach of children and pets.

- Keep packaging material away from children (risk of suffocation).


- Please note the general hygiene measures when using the insect bite healer.

- If the device does not work properly, or if you feel unwell or experience pain, stop using it immediately.

- If users or patients have occurred any serious incident that relation to the device, please report to manufacturer and the competent authority of the Member State in which you are established.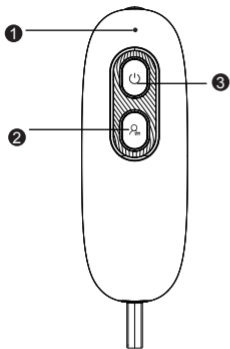

- ① Device status LED
- ② Activation button
- ③ Power button

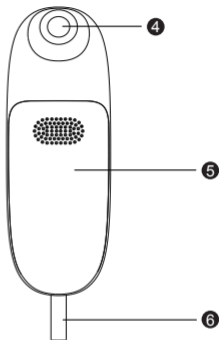- ④ Heating plate(MCH)
- ⑤ Battery compartment lid
- ⑥ Silicone rope


## Power Connection

Inserting the batteries

1. Push open the battery compartment lid **5**.
2. Insert the batteries included in delivery into the battery compartment as shown inside the battery compartment. When inserting the batteries, ensure that the correct polarity (+/-) is observed.
3. Slide the battery compartment lid **5** closed until you feel and hear it click into place. Install the battery as above.

Remove the batteries if you do not intend to use the device for a long time (over 3 months.)

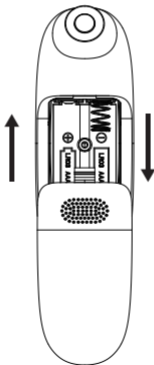# Usage

## WARNING

Before use, make sure that there is no insect sting remaining in the area that was stung. Remove the sting carefully before use. Use immediately after the insect bite or sting achieves the best results. If you wait too long to treat the area that has been stung or bitten and itching and swelling have already started, the insect bite healer can only have a limited effect. Nevertheless, treat the area with the insect bite healer. Usually, itching can be relieved and healing can be sped up.

1. Press the power button **3** to start the device. The device status LED **1** lights up green, an buzzer sounds and indicates to you that the device is ready to use.

Activation Button	Mode	Temperature	Duration	Recommended Use
Single Click	First Mode	47 °C±2°C (116.6°F±3.6°F)	3 seconds	First-time or sensitive skin
Double Click	Second Mode	50 °C±2°C (122°F±3.6°F)	6 seconds	Regular skin

2. Mode 1: After being stung or bitten, apply the heating plate (MCH) **4** of the device to the affected area and press the activation button **2** once. The device status LED **1** will illuminate blue, indicating that the application has started. The device maintains a treatment temperature of 47°C ±2°C (116.6°F ±3.6°F) for 3 seconds. This short application time is recommended for first use or use on sensitive skin.

3. Mode 2: After being stung or bitten, apply the heating plate (MCH) ④ of the device to the affected area and press the activation button ② twice in quick succession. The device status LED ① will illuminate yellow, indicating that the application has started. The device maintains a treatment temperature of 50°C ±2°C (122°F ±3.6°F) for 6 seconds. This application time is recommended for regular use on normal skin.

4. After the end of the application, a buzzer sounds, heating of the heating plate (MCH) ④ stops automatically and the device status LED ① lights up green again. Remove the heating plate (MCH) ④ from the area of skin once the signal sounds.

If you still have itching after the above operation, you can repeat the above operation, but please note that you should not stimulate the same area more than 5 times within one hour. If your itching still cannot be solved, please go to the hospital for medical treatment if necessary.

## **IMPORTANT**

Depending on the sensitivity of the person using the device and of the area of skin, the treatment temperature may feel unpleasant and cause slight skin redness and, in rare cases, irritated skin.

To achieve this itch-relieving effect on the skin nerves, a short period of heat pain is necessary. Only mild heat pain is required for treatment success. If the heat function feels too hot, stop treatment immediately.

5. If required, you can use the heat function of the insect bite healer on the

sting/bite again after a break of 2 minutes or also use it immediately on another sting/bite in a different area. The maximum number of 5 applications per hour on the same treatment area must not be exceeded.

6. If the device is no longer required, press the power button ③ to turn off the device. The green device status LED ① turns off. If there is no further operation within 5 minutes, the device will automatically shut down.

7. Before storing, make sure that the device is switched off and the heating plate ④ has cooled down.

### **Battery warning**

Before the battery level becomes too low for other applications, the device status LED ① starts to flash 3x and an acoustic signal sounds 3x. Replace both batteries soon.

## **Safety Notes**

- If your skin or eyes come into contact with battery fluid, rinse the affected areas with water and seek medical assistance.
- Choking hazard! Small children may swallow and choke on batteries. Therefore, store batteries out of the reach of small children.
- Risk of explosion! Do not throw batteries into a fire.
- If a battery has leaked, put on protective gloves and clean the battery compartment with a dry cloth.
- Do not disassemble, open or crush the batteries.
- Observe the plus (+) and minus (-) polarity signs.
- Protect the batteries from excessive heat.

- Do not charge or short-circuit batteries.
- If the device is not to be used for a relatively long period, take the batteries out of the battery compartment.
- Use identical or equivalent battery types only.
- Always replace all batteries at the same time.
- Do not use rechargeable batteries! Note on electromagnetic compatibility
- The device is suitable for use in all environments listed in these instructions for use, including domestic environments.
- The use of the device may be limited in the presence of electromagnetic disturbances. This could result in issues such as error messages or the failure of the device.
- Avoid using this device directly next to other devices or stacked on top of other devices, as this could lead to faulty operation. If, however, it is necessary to use the device in the manner stated above, this device as well as the other devices must be monitored to ensure they are working properly.
- Failure to comply with the above can impair the performance of the device.

## Explanation of Symbols

The following symbols are used on the device, in these instructions for use, on the packaging and on the type plate for the device:

	Caution
	Refer to instructions manual

	Type BF Applied part
	Disposal in accordance with the Waste Electrical and Electronic Equipment EC Directive – WEEE
	Manufacturer
	Date of Manufacture
<b>LOT</b>	Batch code
IP42	4 Protected against solid foreign objects of 1,0 mm Ø and greater 2 Protected against vertically falling water drops when enclosure tilted up to 15°
<b>EC REP</b>	Authorized representative in the European Community
	CE mark
<b>RoHS</b>	RoHS mark
<b>MD</b>	Medical device
	Indicates the entity importing the medical device into the local
<b>UDI</b>	Unique device identifier

## General Precautions

- Do not touch the device with wet hands. Do not allow water to spray onto the device. Only operate the device if it is completely dry.
- Do not use any detergents or solvents to clean and maintain the device. If liquid penetrates the device this could cause damage to the electronics or other device parts and lead to a malfunction.
- The device should only be used by one person; use by multiple people is not recommended.
- Protect the device against direct sunlight and store in a dry place.
- Protect the device from impacts, moisture, dust, chemicals, marked temperature fluctuations and nearby sources of heat (ovens, heaters).
- Ensure that no water enters the device. If this should happen, remove the batteries and only use the device again when it is completely dry.
- If you store the device, no heavy objects should be placed on top of it.
- Before use, ensure that there is no visible damage to the device and karabiner. In case of any doubt, do not use the device, and contact your retailer or the specified Customer Service address.
- Store the device in a location protected against climatic influences. The device must be stored in the environmental conditions specified.
- When ambient temperature is less than  $-10^{\circ}\text{C}$  or higher than  $50^{\circ}\text{C}$  , The time from minimum storage temperature to use is at least 4 hour; The time from maximum storage temperature is at least 4 hour.

## What If There Are Problems?

Problem	Possible cause	Solution
The device does not switch on.	The battery installation is incorrect.	Install the battery over again.
	The batteries are flat.	Replace both batteries as described in these instructions for use.
The device no longer gets hot enough.	The battery level is too low.	Replace both batteries as described in these instructions for use.
Device outputs 3 acoustic signals/the device status LED ❶ flashes – even after switching the device OFF and ON again.	Battery level is too low, meaning that the device may no longer be able to be operated.	Replace both batteries as described in these instructions for use.
Application cannot be started, even though the device status LED ❶ lights up green.	The device needs to be restarted.	Press the power button ❸ again.
Device outputs 5 acoustic signals/the device status LED ❶ flashes – even after switching the device OFF and ON again.	The device may be faulty.	Please contact Customer Services.

# Cleaning and Care

- The device is intended for multiple uses.
- It is recommended that the device is cleaned before first use and each additional use.
- The device must be switched off, allowed to cool down and the batteries removed each time before cleaning.
- Use only soft, dry cloths for cleaning the housing.
- Never use any substances for cleaning that could potentially be toxic if they came into contact with the skin or mucous membranes, or if they were swallowed or inhaled.
- Do not use any abrasive cleaning products and never submerge the device in water.
- Do not clean the device and karabiner in the dishwasher!
- Avoid mechanical cleaning with brushes or similar, as this could result in irreparable damage.
- When using the device on multiple areas that have been stung and bitten, it is recommended that you clean the heating plate each time. This reduces the possibility of germs and bacteria being transferred from one area of skin to the next. Use a commercially available alcohol disinfection wipe to disinfect the heating plate.

## Disposal

This product must not be disposed of together with domestic waste.

All users are obliged to hand in all electrical or electronic devices, regardless of whether or not they contain toxic substances, at a municipal or commercial collection point so that they can be disposed of in an environmentally acceptable manner. Remove the batteries before disposing of the device/unit. Do not dispose of old batteries with your household waste, but at a battery collection station at a recycling site or in a shop.

Consult your local authority or your supplier for information about disposal.

# Technical Specifications

Type	IBR30
Model	IBR30
Dimensions (L x W x H)	109.5mm×34mm×24.2mm
Weight (without batteries)	25g
Power supply	2 x batteries,type AAA(LR03 1.5V)
Max. temperature	50°C ±2°C (122°F ±3.6°F)
Functions	sensitive application: 3s,47°C±2°C(116.6°F±3.6°F) regular application: 6s,50°C±2°C (122°F±3.6°F)
Service life	5 years
Expected operating duration	Approx. 300 applications (per batteries)
Permissible storage and transport environment	-10°C to 50°C, relative humidity 30% to 85%, pressure 700hPa to 1060hPa
Permissible operating environment	10°C to 40°C, relative humidity 30% to 85%, pressure 700hPa to1060hPa

## Essential Performance:

Temperature control, Max temperature is 50°C ±2°C.

\* in case of longer storage without batteries (please remove the batteries and store them cool < 30°C)

**In accordance with our policy of continual product improvement, we reserve the right to make technical and visual changes without notice.**

# Warranty and Repair Terms

Please contact your dealer or the service centre in case of a claim under the warranty. If you have to return the unit, please enclose a copy of your receipt and state what the defect is. The following warranty terms apply:

1. The warranty period for products is one years from date of purchase. In case of a warranty claim, the date of purchase has to be proven by means of the sales receipt or invoice.
2. Defects in material or workmanship will be removed free of charge within the warranty period.
3. Repairs under warranty do not extend the warranty period either for the unit or for the replacement parts.
4. The following is excluded under the warranty:
  - a. All damage which has arisen due to improper treatment, e.g. nonobservance of the user instructions.
  - b. All damage which is due to repairs or tampering by the customer or unauthorised third parties.
  - c. Damage which has arisen during transport from the manufacturer to the consumer or during transport to the service centre.
  - d. Accessories which are subject to normal wear and tear.
5. Liability for direct or indirect consequential losses caused by the unit are excluded even if the damage to the unit is accepted as a warranty claim.

## **EMC Information**

The ME EQUIPMENT or ME SYSTEM is suitable for home healthcare environments.

Warning: Don't near active HF surgical equipment and the RF shielded room of an ME system for magnetic resonance imaging, where the intensity of EM disturbances is high.

Warning: Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.

Warning: Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.”

Warning: Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the equipment, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.

The insect bite healer IBR30 may be interfered with by other equipment, even if that equipment complies emission requirements.

# Electromagnetic Emissions

Guidance and manufacturer's statement - Electromagnetic emission	
Emission test	Compliance
RF emissions CISPR 11	Group 1
RF emissions CISPR 11	Class B
Harmonic emissions IEC 61000-3-2	Not applicable
Voltage fluctuations/ flicker emissions IEC 61000-3-3	Not applicable

# Electromagnetic Immunity

Guidance and manufacturer's declaration - Electromagnetic Immunity		
Immunity Test	IEC 60601-1-2 Test level	Compliance level
Electrostatic discharge (ESD) IEC 61000-4-2	$\pm 8$ kV contact $\pm 2$ kV, $\pm 4$ kV, $\pm 8$ kV, $\pm 15$ kV air	$\pm 8$ kV contact $\pm 2$ kV, $\pm 4$ kV, $\pm 8$ kV, $\pm 15$ kV air
Electrical fast transient/burst IEC 61000-4-4	Not applicable	Not applicable
Surge IEC 61000-4-5	Not applicable	Not applicable
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	Not applicable	Not applicable
Power frequency magnetic field IEC 61000-4-8	30 A/m 50Hz/60Hz	30 A/m 50Hz/60Hz
Conducted RF IEC61000-4-6	Not applicable	Not applicable
Radiated RF IEC61000-4-3	10 V/m 80 MHz – 2,7 GHz 80 % AM at 1 kHz	10 V/m 80 MHz – 2,7 GHz 80 % AM at 1 kHz
NOTE UT is the a.c. mains voltage prior to application of the test level.		

# Electromagnetic Immunity

## Guidance and manufacturer's declaration - Electromagnetic Immunity

	Test Frequency (MHz)	Band (MHz)	Service	Modulation	Maximum Power (W)	Distance (m)	IEC 60601-1-2 Test Level (V/m)	Compliance level (V/m)
Radiated RF IEC61000-4-3 (Test specifications for ENCLOSURE PORT IMMUNITY to RF wireless communications equipment)	385	380-390	TETRA 400	Pulse modulation 18 Hz	1.8	0.3	27	27
	400	430-470	GMRS 460 FRS 460	FM $\pm$ 5 kHz deviation 1 kHz sine	2	0.3	28	28
	710	704 - 787	LTE Band 13,17	Pulse modulation 217 Hz	0.2	0.3	9	9
	745							
	780							
	810	800 - 960	GSM 800/900, TETRA 800, iDEN 820, CDMA 850, LTE Band 5	Pulse modulation 18 Hz	2	0.3	28	28
	870							
	930							
	810	1700-1990	GSM 1800; CDMA 1900; GSM 1900; DECT; LTE Band 1, 3,4, 25; UMTS	Pulse modulation 217 Hz	2	0.3	28	28
	870							
	930							
	2450	2400-2570	Bluetooth, WLAN, 802.11 b/g/n, RFID 2450, LTE Band 7	Pulse modulation 217 Hz	2	0.3	28	28
	810	5100-5800	WLAN 802.11a/n	Pulse modulation 217 Hz	0.2	0.3	9	9
	870							
	930							

## Guidance and manufacturer's declaration - electromagnetic Immunity

Radiated RF IEC61000-4-39 (Test specifications for	Test Frequency	Modulation	IEC 60601-1-2 Test Level (A/m)	Compliance level (A/m)
ENCLOSURE PORT IMMUNITY to proximity magnetic fields)	30 kHz	CW	8	8
	134.2 kHz	Pulse modulation 2.1 kHz	65	65
	13.56 kHz	Pulse modulation 50 kHz	7.5	7.5

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Share Info GmbH
AOJ HEALTH Technology Co.,Ltd