AT52329

**WÓZEK INWALIDZKI, ELEKTRYCZNY INSTRUKCJA OBSŁUGI
ELECTRIC WHEELCHAIR USER MANUAL
ELEKTOROLLSTUHL GEBRAUCHSANWEISUNG
UŽIVATELSKÁ PŘÍRUČKA K ELEKTRICKÉMU INVALIDNÍMU VOZÍKU
NÁVOD NA POUŽITIE ELEKTRICKÝ INVALIDNÝ VOZÍK
MANUEL D'UTILISATION DU FAUTEUIL ROULANT ÉLECTRIQUE
GEBRUIKERSHANDLEIDING ELEKTRISCHE ROLSTOEL
MANUAL DE USUARIO DE LA SILLA DE RUEDAS ELÉCTRICA
MANUALE D'USO DELLA SEDIA A ROTELLE ELETTRICA
ANVÄNDARHANDBOK FÖR ELEKTRISK RULLSTOL**

MD

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EN

Thank you for purchasing our product, please read the instructions for use carefully. Do not use the product before reading and understanding this manual. If you do not understand the warnings, cautions and recommendations, please contact your health care professional or dealer to avoid damage to the product.

NOTE: Inspect all parts for damage caused in transit. If you notice such damage, DO NOT use the product. Contact the manufacturer for more information.

TRANSFER

The electric wheelchair is a motorized vehicle for indoor and outdoor use. Suitable for people with mobility difficulties and the elderly.

FEATURES

- Comfortable cushion and backrest
- New smarter and safer engine
- Lithium battery with operating power monitoring function
- Operation of manual mode and joystick mode

ELECTRICAL SAFETY

- Classification of protection against electric shock: Class I internal electrical supply
- Degree of protection against electric shock: B
- Protection against liquid ingress: IPX4
- Work classification: continuous work
- Rated voltage and frequency: input voltage AC110V-220V, frequency 50 Hz 60 Hz, internal supply voltage DC 24V
- Input power: 250VA

TECHNICAL PARAMETERS

Dimensions when folded (w.d.h.): 63x106x60 cm

Length: 112 - 177 cm +/-1 cm

Width: 63 cm +/-1 cm

Height: 101-127 cm +/-1 cm

Seat width: 43 cm +/-1 cm

Seat depth: 45 cm +/-1 cm

Clearance of the seat relative to the ground: 52.5 mm

Armrest height: 18.5 cm +/- 1 cm

Backrest height: 49 cm

Top speed: 6 +/-0.5 km/h

Braking on a flat road: ≤1.5m

Slope braking: ≤2 m (6°)

Static stability: 9°

Dynamic stability: 6°

Maximum elevation angle: 6°

Maximum obstacle height: 4 cm

Maximum width of cross ditch: 10 cm

Minimum spin radius: 80 cm

Size of the front multiwheel: 8 inches

Rear wheel size: 10 inches

Type of wheels: solid

Braking system: automatic

Shock absorber: hydraulic

Total weight: 73.5 kg

Maximum load: 100 kg

Number of batteries: 1

Headlight: White LED light*2

Relative humidity: <80%

Operating temperature: -20°C-45°C

Atmospheric pressure: 860hPa - 1060hPa

Internal power supply voltage: DC 24V

Electrical parameters

Motor voltage rating: 24V

Rated motor power 250W*2

Battery parameters

Battery type Lithium battery


Rated voltage: DC25.6 V

Rated capacity: 25Ah

Total charging time: 3,5 h

Range: 25 km

Charging temperature: 0-45°C

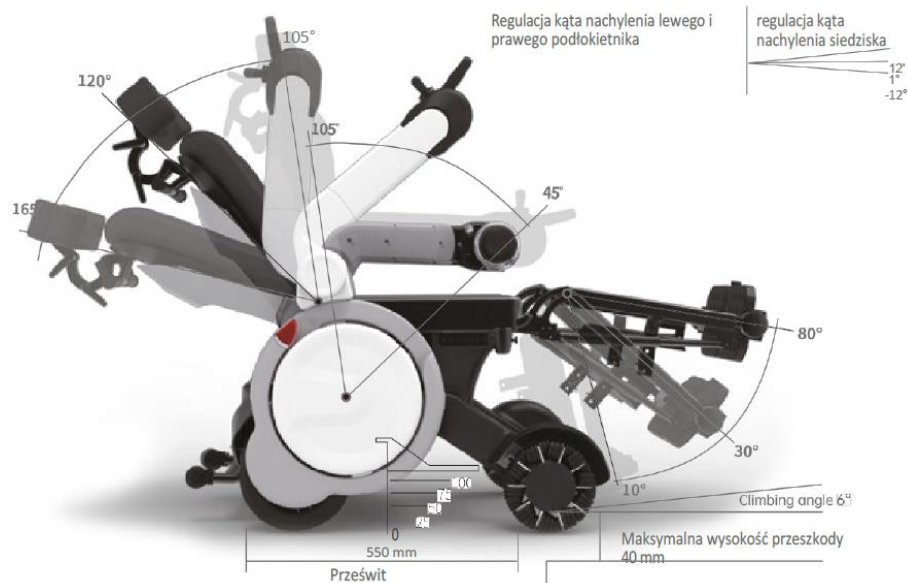NOTES

- Follow the instructions in this manual.

Depending on the user's weight, seat height (the height of the upper body on the chair) and position sitting, tilt friction and seat support will be affected. Adjustment of parameters should be discussed with a doctor or specialist.

- When driving outdoors or on non-motorized roads, observe all

Traffic regulations for non-motorized vehicles;

- Do not lean on the lock button! This may cause the product to stop working during operation.

- For safety reasons, do not exceed the recommended load limit for wheelchairs.

Do not stand on the footrests, it can lead to a dangerous accident, injury to the user or the product.

- Do not stand on the system to prevent tipping.

- Do not use the product under conditions beyond the limits described in this manual.

If the operation of this product is abnormal, stop using it and contact your local dealer or with our customer service department. Do not connect electrical and mechanical devices to Products that have not been approved by our company.

Failure to follow the instructions may result in personal injury and void the product warranty.

NOTES ON USE

- Before driving, make sure you are seated and your seat belt is fastened and in good condition.

- Use the product in a planned manner. Before driving, make sure the product is fully charged and the target distance is within range.

- The speed should be adjusted according to the user's driving skills and the driving environment. It is recommended to set the minimum speed before getting used to driving the product. In addition, it is strongly recommended to set the minimum speed for indoor use.

- Remember to slow down when approaching areas of poor visibility, sharp turns, sharp curves and many obstacles.

- If a small obstacle is encountered, the device may get stuck and be unable to move; therefore, make sure there are no obstacles in the way. Carelessness may cause injury or damage to objects.

- In case of an emergency stop, release the joystick or point it in the opposite direction.

- Before stepping down from the wheelchair, turn off the power button and remove the key.

- When bypassing an obstacle, make sure the height of the obstacle is less than 40 mm.

- Don't pass if you don't feel confident on a sloping road.

- When using the product for the first time, drive on a flat and hard surface, and make sure that the w

There are no obstacles or pedestrians nearby.

- This product can only be used by one person.

- When driving, make sure that the slope of the road does not exceed 6°.

- In a sloping area, first check that the substrate is not wet, snowy, sticky, etc. Besides, due to the difference in the coefficient of friction of the substrate, such as paint and flooring materials, it will be easy to slip, so pay attention to this.

- When going down a hill, to avoid excessive stopping distance, drive at the lowest constant speed

- Reducing speed during a descent can increase friction, which increases maneuverability and safety

- Do not park on slippery roads, on inclines or on the edges of roads. When driving, pay attention to

Attention to slopes and curbs.

- When riding on this product, keep your center of gravity balanced, do not extend your arms, twist your body or lean against the seat back.

- Do not use the device on the highway. When crossing the road, make sure you are within sight of nearby drivers.

- For safety reasons and in case of failure of this product, do not drive on snowy and icy roads

- Do not drive in water more than 100mm deep. Failure to do so may result in internal defects.

- When in use, avoid wearing a raincoat, long skirt and other clothing that can wrap around the front and rear wheels. Wrapping clothing around the front and rear wheels can not only lead to a dangerous accident.

DESCRIPTION OF THE ELEMENTS

- | | |
|-------------------|-------------------------|
| 1. Headrest | 14. rear cover |
| 2. Support | 15. direction blinker |
| 3. Sidecover | 16. battery |
| 4. Rearwheel | 17. clutch |
| 5. Front wheel | 18. safety cover |
| 6. Footrestplates | 19. charging socket |
| 7. Leg rest | 20. anti-tipping system |
| 8. Footrests | 21. key |
| 9. Seat | 22. joystick functions |
| 10. Headlight | |
| 11. USB port | |
| 12. Joystick | |
| 13. Armrest | |

JOYSTICK

- | |
|---|
| 1. Handle |
| 2. Power button |
| 3. Increase in speed |
| 4. Right turnsignal/ Right footrest |
| 5. Indicator of unfolding function of elements wheelchair |
| 6. Light |
| 7. Speed and code display |
| 8. Range |
| 9. Battery charge status |
| 10. Leftturnsignal/ Leftfootrest |
| 11. Speedreduction |
| 12. Features |
| 13. Horn |

SAFETY CHECK BEFORE EACH USE

- Unplug the charger and place it in a safe place before turning on the power.
- Do not step on the power cord or charger connector. If the wires break or become entangled, it may cause electrical shock to the user.
- Make sure the front and rear tires have no signs of cuts, cracks, deformation or other damage.
- Make sure the product is free of physical damage and corrosion, such as dents and scratches.
- Confirm the state of charge of the battery.

SUPPLY

- The battery consumption of this product is represented by LEDs. Please refer to the following figure for detailed information.
- If you find that the power consumption of this product is too fast or the LED light displays incorrect power, contact your local dealer or consult customer service.
- The maximum range of this is 25 km. Actual range will vary due to slope, curves, terrain, driving habits, load, temperature and other factors.

Note: When the battery level indicates 2 dashes, charge the battery. 5 dashes indicates full charge.

NOTE: Fully charge the battery before using the product for the first time. Actual distance may vary due to slope, curves, terrain, driving habits, load, temperature and other factors.

To achieve long-distance driving, it is recommended to do the following:

- Make sure the battery is fully charged before using the wheelchair.
- The maximum load of the product must not be exceeded
- Avoid hills and obstacles on the road;
- Drive at a constant speed and avoid sudden and frequent braking. When the charge indicator is lower than 2 dashes, recharge the battery.
- The battery charger has an overcharge protection function that will not cause damage to the battery even when charged for more than 12 hours. For safety reasons, however, do not leave the power supply connected to this product for so long.
- Batteries should not be charged outdoors.
- Do not charge the battery with the clutch unlocked.
- Do not use an AC outlet without a ground terminal for charging.
- Do not charge the product near a fire source.
- Do not use any mains chargers other than our company's original chargers.
- Do not cut the ground plug of the battery charger cable with the battery power switch or modify the power switch and charging system.
- Read the battery and charging instructions carefully and follow them.
- Failure to follow the instructions may result in fire, explosion or accidental injury.

LOADING METHOD

- First, park the wheelchair in a well-ventilated room (charging environment temperature 0-45°C).
 - Turn off the power.
 - Plug the power adapter plug into the charging socket on the back of the wheelchair; plug the input plug into the socket.
 - When the battery is charged, the LED on the power switch glows orange and the power indicator column is in the operating state.
 - When the LED on the adapter turns green, it means that the battery is fully charged.F.
 - You can check the battery level with the battery level indicator.
- (When the power supply is plugged into an extension cord outlet, the LED on the power supply glows green.
When the power supply is connected to the wheelchair charging socket, the LED on the power supply lights up orange).

BATTERY MAINTENANCE

- The battery life of this product can reach about 1,500 charges.
- When the performance of the battery drops and the time of use after each charge is abnormally short, it means, That battery life is nearing its end.
- The battery may only be repaired and replaced by qualified professionals. Users are not allowed to replace the battery themselves.
- If you think the battery needs to be replaced, contact your local dealer or our company to Consultation.

TRANSPORT AND STORAGE ENVIRONMENT

Transportation and storage temperature: -20C +50C
Transportation and storage humidity: < 80%
Atmospheric pressure: 860hPa - 1060hPa

PRODUCT MAINTENANCE

- Regularly check that the wheels of the wheelchair are not loose
- Regularly check that the seat is not loose
- Regularly check that the anti-tip wheels are not loose.
- You should regularly check that the clutch lever meets the braking requirements. You can park the wheelchair on an incline and observe that the wheel of the wheelchair does not slip. It is recommended once every six months.
- Before driving, check the tire wear level. If the tire is worn to the limit of the wheel tread, replace it with a new tire immediately.

CLEANING AND MAINTENANCE OF PRODUCTS

- Wipe the case of this product with clean water, avoid placing the product in a damp place; protect the joystick from impact when transporting this product.

If the joystick and control parts become contaminated with food or beverages, clean them immediately,
Wipe with a cloth moistened with a mild cleaning solution, avoid using cleaning agents,
Such as alcohol.

- Seat and backrest covers: Clean seat and backrest covers with a wet soapy cloth and then wipe dry.
- General maintenance: from time to time, check that the exposed screws of this product are securely fastened.

STORAGE

- This product should be stored in a dry place when not in use, and should never be left on a damp surface.

- If the product is not used for a long time, protect it from dust and fully charge it before storage, and charge the battery at least once a month during storage.

SAFETY WHILE DRIVING

Enabling


- Use the power on/off button to turn on the product.
- Press the power button on the function panel, and the LED display will be on/off.

Shutting down

- When turning off the product, proceed in the following order: press the power button to close the display interface; Then remove the key.
- If the correct shutdown steps do not work, please consult your dealer or our company's customer service department.

SPEED CONTROL

- Speed is adjusted using the decrease/increase speed buttons on the joystick.

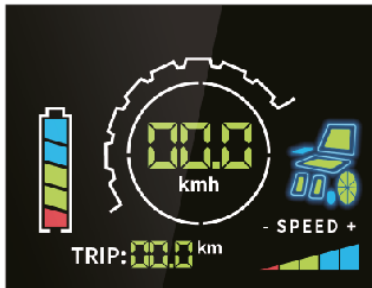1. Maximum speed

2. minimum speed

DIRECTIONAL CONTROL

The direction of travel of this product is controlled by changing the direction of the joystick.

Pushing the joystick forward causes forward movement; pushing the joystick backward causes backward movement.

When the joystick is pressed to the left, the product rotates to the left; when the joystick is pressed to the right, the wheelchair rotates to the right.

CONFIRMATION OF SAFETY CONDITIONS BEFORE USE

Before using this product, make sure it is handled and driven on a flat place without obstruction, accompanied by a support staff.

- In case of problems, carry a cell phone and other devices to communicate with the outside world. Make sure the power supply to this product is connected before use.

To make sure you can get to your destination, make sure the product has enough power.

For details, see 4.5 Battery power confirmation.

- Slowly and carefully push and pull the joystick in the direction you want to go, keeping the speed switch button on the lowest speeds.

- When driving to a safe place, make sure that the product can move in any direction, keeping the Speed switch button on the lowest speed.

- If you experience unusual sounds or vibrations, please record them and contact your local dealer or our after-sales service.

JOYSTICK CONTROL

1. P-1 backrest adjustment function

Press the „FUNCKJE“ button once, the LED display will switch to P-1 function. At this time, the "backrest" part of the icon on the unfolding function indicator wheelchair components.

When the joystick is pressed, the backrest moves forward. Maximum angle Backrest forward: fold the backrest down completely (do not fold down completely of support when a patient is in a wheelchair).

The backrest moves backward when the joystick is pulled backward.

Maximum angle of backrest tilt backward: about 165° relative to the

seat cushion (the angle of this position is limited).
When you hear a beep, it means that the adjustment has been carried out to the maximum degree and cannot be continued.

2. P-2 seat adjustment function

Press the "Functions" button twice, the LED display will switch to P 2 function, and the "seat" part of the icon on the unfolding function indicator wheelchair components will start flashing.

When the joystick is moved forward, the seat tilts forward. Seat tilts forward at a maximum angle of about 12°. (Do not push the joystick forward when the leg rest touches the floor to avoid damage parts of the footrest).

When the joystick is pulled back, the seat tilts backward. The seat tilts back at a maximum angle of about 24°. (The anti-tipping wheels touch the floors)

- Hearing a beep indicates that the adjustment has been made to the maximum degree and cannot be continued.

3. P-3 leg support adjustment function - double leg support adjustment.

Press the "Functions" button three times, and the LED display switches to the P-3 function,

When the joystick is pushed forward, the leg support will move upward.

The leg rest moves upward at an angle approximately equal to the seat. (Adjusting the direction of the leg support plate and footrest plates can cause the user's leg to be placed horizontally on the leg support.

The leg rest moves downward when the joystick is pulled back. Support under the legs moves downward at an angle of about 75° to the seat.

Single adjustment of leg support.

Press the "function button" three times, the LED display will switch to the P-3 function,

During this time, the dynamic icon in the "leg support" section will flash.

Press the arrow buttons "<" (or ">") again

When the joystick is pushed forward, the corresponding left (or right)

The footrest will move upward. The footrest moves upward under the seat at an angle approximately equal to the seat.

When the joystick is pulled back, the corresponding left (or right) leg support will move downward. The downward movement angle of the leg rest is about 10° from the seat (allowing the user's feet to be placed on the footrest plate).

4. One-touch folding/unfolding function of P-5/P-6.

NOTE: Before operating, remove the leg supports on both sides of the wheelchair.

If the leg supports are not removed, the one-touch folding function will be unavailable, and the controller will turn on the alarm.

-Press the "functions" and "-" buttons at the same time, the LED display will switch to the P 5 function, at this time part of the icon dynamic "backrest" and "seat" will flash at the same time.

The product folds automatically and takes about 25 seconds.

-Press the function buttons "functions" and "+" at the same time, the LED display will switch to function P-6, at this time part of the dynamic "backrest" and "seat" icons will flash at the same time.

The product will automatically unfold and take about 25 seconds.

SETTING THE USER'S HOME POSITION

Folding to the home position: press "function button" + "headlight button" simultaneously until you hear a sound alarm from the controller, and then press the function button, the LED display will switch to the P-1 function.

The "backrest" part of the dynamic icon flashes. When the joystick is pushed forward, the backrest will move forward until it is clinging to the seat.

Press the function button two more times, the LED display switches to the P-2 function, and part of the dynamic symbol "seat" will start flashing. Push the joystick forward.

The seat will move down until it is about 1 cm above the wheel cover. After setting the above two functions turn off the power to save the settings.

Changing the home position: Press the "Function Button" + "Trumpet Button" at the same time until you hear the alarm sound from the controller, and then press the function button, the LED display will switch to the P-1 function.

The "backrest" part of the dynamic icon flashes. Push the joystick backward and the backrest will move backward until the angle is set

Between the backrest and the seat is about 105°.

Press the function button two more times, the LED display will switch to the P-2 function, and part of the dynamic symbol " seat " will start flashing. Press the joystick backward.

The seat will move upward until it reaches a horizontal position. After setting the above two functions, turn off power to save the settings.

warning

- Do not turn off or turn off the power during the process of setting the home position, otherwise the position will be defaulted to set as the starting position.

- The starting position is the position of the backrest or seat when the wheelchair is unfolded or folded down

PARKING

- Sit still and hold on to the armrest while braking. Release the joystick and the electric wheelchair will stop in a few seconds.

- If you need to stop, just turn off the switch.

- Do not get in or out of the wheelchair with the power on to avoid moving the joystick



With their own body or clothing causing the wheelchair to move.

Instructions on how to clutch and change the wheelchair mode to manual

a) The clutch is locked, and two switches are raised simultaneously to use the joystick to steer the wheelchair;

b) The clutch must be unlocked and the two switches are pressed at the same time to control the carriage manually (the joystick must not be used in this mode)

Note: Do not change mode on sloping surfaces.

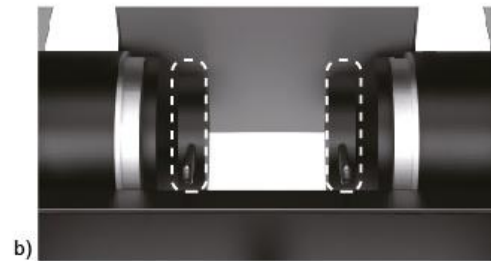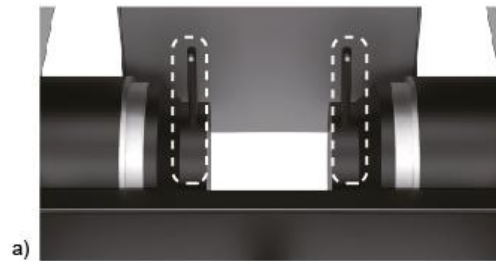ANTI-TIPPING SYSTEM

The anti-tilt feature prevents the product from tipping backwards during upward and downward travel Down the slope. Do not remove the safety device during use.

USB PORT INSTRUCTIONS

- The USB interface of this product is only for charging external cell phones and other devices. Output voltage: DC 5V.

BLUETOOTH CONTROL

- Turn on Bluetooth on your phone and search for "If Health," then pair your device.

- Download the IF HEALTH app from the app store. By connecting to the app, the wheelchair can be controlled from the phone's position.

- Joystick mode

Press the center dot to simulate joystick operation. Release the dot to stop.

- Button mode


Press the arrow buttons to control direction and travel.

- Speed control

Press the "speed + " and "speed - " buttons to adjust the speed

Press the " speed + " button to increase the speed of the wheelchair.

Press the " speed - " button to reduce the speed of the wheelchair.

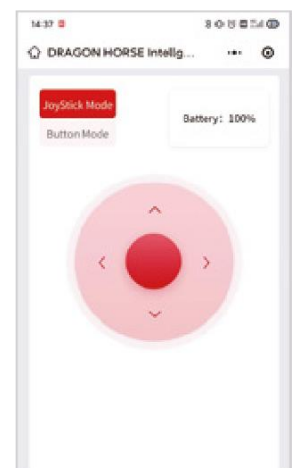- One-touch folding function

To use this function, remove the footrests on both sides of the wheelchair.

A long press on the folding button automatically

folding the wheelchair; long press on the unfolding button causes the wheelchair to automatically unfold.

NOTES:

In the event of a product-related "serious incident" that directly or indirectly led to, may have led to, or may lead to any of the following:

(a) death of a patient, user or other person or

(b) temporary or permanent deterioration of the health of the patient, user or other person or

(c) a serious threat to public health

the above "serious incident" should be reported to the manufacturer and to the competent authority of the Member State in which the user or patient resides. In the case of Poland, the competent authority is the Office for Registration of Medicinal Products, Medical Devices and Biocidal Products.

NOTES:

If you experience pain, allergic reactions, or other disturbing symptoms that are unclear to you related to the use of the medical device, consult a healthcare professional.

CONTRAINDICATIONS

Physical or mental limitations (e.g., visual impairment) that prevent safe handling.

- If a person has difficulty with motor coordination or is unable to operate the wheelchair's controls independently, this may be a contraindication.

- People with disorders of consciousness that may affect their ability to safely operate a wheelchair may also be contraindicated.

- In some cases, serious heart or blood pressure problems may limit the ability to safely use an electric wheelchair, especially when emotions or stress may trigger adverse reactions.

- Conditions such as advanced multiple sclerosis or other diseases that affect muscle control may require specialized adaptations in wheelchairs.

- Problems with depth perception, space, or orientation in general can significantly limit the safety of using an electric wheelchair.

- General malaise or recent surgeries, especially in the spine or lower extremities, may require withholding the use of a wheelchair until the condition improves.

NOTE: it is forbidden to use the product other than for its intended purpose .

NOTE: The manufacturer is not responsible for damage caused by negligent cleaning and maintenance, or resulting from failure to follow the recommendations in this manual.

TARGET PATIENT GROUP

Persons struggling with diseases, dysfunctions or injuries, for the treatment/rehabilitation or compensation of which this device is intended (see the section on the purpose of the device of this manual). The device may be purchased by the user independently or on the recommendation of a physician, therapist or other specialist. Whether purchasing the device on its own or as indicated by a doctor/therapist/other specialist, the available sizes/necessary functions and variants of the device, indications and contraindications for use, as well as information provided by the manufacturer, should be taken into account.

METHOD OF DISPOSAL OF THE PRODUCT

Dispose of the electrical medical device by following the applicable rules for the disposal of electrical equipment.

GUARANTEE

Frame: 12 months

Battery: 6 months

PROBLEM SOLVING

TROUBLE			
	<p>Looped sound intermittently, displays "1" on screen</p>	<p>Battery level too low</p>	<p>1. Before turning on the truck, you should charge it for more than 30 minutes. 2. the battery must be returned to factories for repair.</p>
	<p>Two short looped beeps, the screen displays "2"</p>	<p>Left engine failure</p>	<p>Restart the wheelchair, if it does not help the wheelchair must be sent back to the Service for repair.</p>
	<p>Three short looped beeps, the screen displays "3"</p>	<p>Left brake failure</p>	<p>Restart the wheelchair, if it does not help the wheelchair must be sent back to the Service for repair.</p>
	<p>Four short looped beeps, the screen displays "4"</p>	<p>Failure of the right engine</p>	<p>Restart the wheelchair, if it does not help the wheelchair must be sent back to the Service for repair.</p>
	<p>Five short looped beeps, the screen displays "5"</p>	<p>Failure of the right brake</p>	<p>Restart the wheelchair, if it does not help the wheelchair must be sent back to the Service for repair.</p>
<p>X</p>	<p>Continuous sound</p>	<p>Joystick failure</p>	<p>Check the position of the joystick, if it is in the correct position the joystick should be sent back for repair</p>
	<p>Eight short looped beeps, the screen displays "8"</p>	<p>Internal controller error</p>	<p>Return the controller to the service center for repairs</p>

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Model:
Serial number:

Point of sale stamp

and a legible signature of the Seller:

1. ANTAR Sp. J., 03-068 Warsaw, ul. Zawisłańska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to ANTAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

ANTAR Sp. J.
03-068 Warsaw, ul. Zawisłańska 43
Tel. 22 518 36 00, Fax 22 518 36 30
www.antar.net

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