Ergonomic Lumbar Cushion

Ergonomisches Lendenkissen

DE

EN

ES

FR

IT

Instructions of use

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Healthfix GmbH
Bentheimer Str. 66
48529 Nordhorn
Germany

REF

Art. 100 401

MD

LENDENKISSEN

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DE

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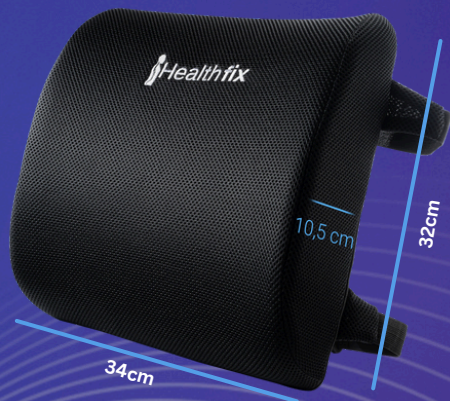
EN

Intended use:

The Healthfix® ergonomic lumbar cushion is used to support and relieve the lumbar spine. It promotes correct sitting posture, relieves back pain and ensures a comfortable sitting experience, especially when sitting for long periods. Ideal for office chairs, car seats and other seating furniture.

Indications:

Chronic back pain
Post-operative recovery
Poor sitting posture
Prevention
Pregnancy

Contraindications

Not to be used for
Acute spinal injuries.
If you have a recent injury or illness in the lumbar region, or if you experience pain when using the lumbar cushion, you should consult a doctor.

Instructions for use:

Place the lumbar cushion on the back of your chair or seat.
Make sure the cushion is positioned at the height of your lumbar spine.
Lean back so that your back is in direct contact with the cushion.
If necessary, adjust the position of the cushion for maximum comfort.

After just 30 seconds of using the Healthfix® lumbar cushion, the MEMORY-FOAM adapts to your body shape with the help of your own body heat.

Intended useful life

- 5 years

Storage and transport conditions:

Constant conditions, ideally with temperatures between 18 and 25 degrees and humidity between 35 and 65%.

Cleaning instructions:

Open the zipper on the side and remove the foam core. The foam core should not be washed as it may lose its effectiveness.

Wash the seat cover in the washing machine at 40°.

Scope of delivery:

1x Healthfix® lumbar cushion
1x mesh cover
Instruction manual and brochure

Warranty and customer service

The Healthfix® lumbar cushion comes with a 12-month warranty.

If you have any questions or problems, please contact our customer service at the telephone number or email address provided.

Safety instructions:

Keep cushions away from sources of heat, sparks or open flames.

Use the cushion only as described in these instructions.

Ensure that the cushion is secure and does not slip.

The cushion is not a toy. Keep the cushion out of the reach of children.

Instructions for safe disposal:

Disposal as plastic waste

Further information on use:

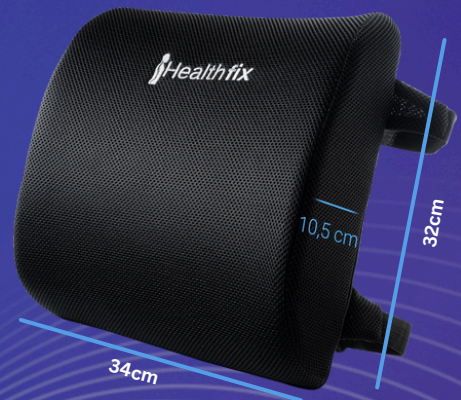
If you have any difficulties with use, please contact your retailer or the responsible Healthfix® customer advisor.

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All rights reserved.
Healthfix GmbH | Bentheimer Str. 66 | 48529 Nordhorn | Germany