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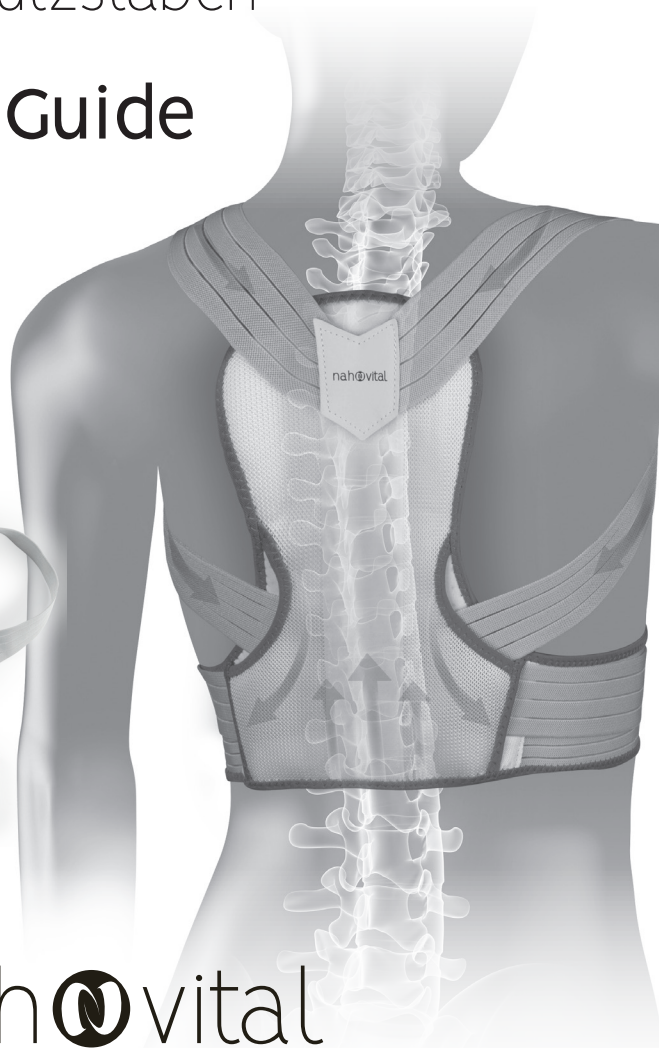
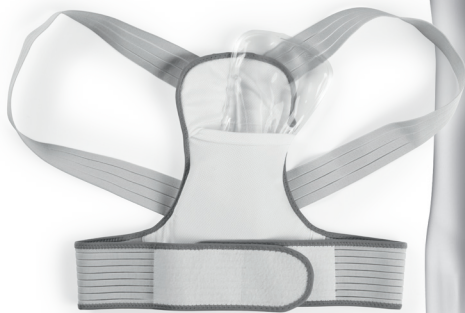
## Guide

nah@vital  
just. care.

**DE**

ab Seite 4

**EN**

from page 8

**FR**

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**ES**

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**IT**

da pagina 20

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## Dear Customer,

We are delighted that you have chosen the **nah-vital posture corrector**. Mobility is crucial in shaping our lives. A straight posture not only has an impact on your health, but also on how we feel about our body and the way we look. Targeted muscle exercises for the back are therefore highly recommended. However, ultimately everyone must remind themselves to adopt a straighter posture every single day.

With the **nah-vital posture corrector** you can remind yourself to adopt a straighter posture. The elastic shoulder straps can be individually adjusted to your back and fixed in place with the hook-and-loop fastener. The soft material nestles perfectly against your skin and is even invisible under your clothing. In addition, you can treat your back however you want with the gel pad: it can be cooled or heated and can easily be inserted in the pocket which is provided for it. The stabilising rods give you additional support and mobility in the spinal area.

If you have any questions about the product and about spare parts/accessories, contact the customer service department via our website: **[www.nah-vital.de](http://www.nah-vital.de)**

We hope you have a lot of fun with your **nah-vital posture corrector**.

## Information About the Operating Instructions

Before using the product for the first time, please read through these operating instructions carefully and keep them for future reference and other users. They form an integral part of the product. The manufacturer and importer do not accept any liability if the information in these operating instructions is not complied with.

---

### SYMBOLS

Danger symbol: This symbol indicates possible dangers. Read the associated safety notices carefully and follow them.

40°C coloured wash

Do not bleach

Do not dry in the tumble dryer

Do not iron

Do not dry-clean

### SIGNAL WORDS

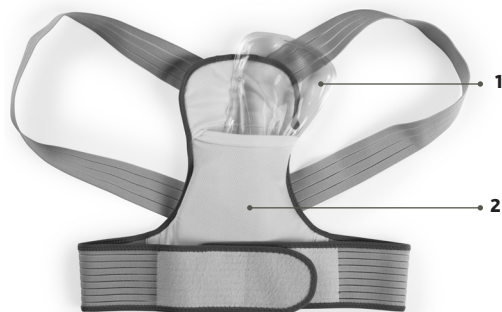
Signal words indicate hazards if the associated notices are not followed.

**WARNING** – medium risk, may result in serious injury or death

**NOTICE** – may result in risk of damage to material

## ITEMS SUPPLIED

- 1 Gel pad, 1x
  - 2 Posture corrector, 1x
- Operating instructions, 1 x

1 Check the items supplied for completeness and the components for transport damage. If you find any damage, do not use the product but contact our customer service department.

Remove any possible films, stickers or transport protection from the product.

## INTENDED USE

- This product is intended to be used to provide short-term support for the back.
- The product is not designed for medical or therapeutic purposes. It is not a substitute for medical treatment. If complaints persist, consult a doctor.
- The gel pad can be used as a cooling pad or a thermal support.
- The product is for personal use only and is not intended for commercial applications.
- Use the product only for the specified purpose and as described in the operating instructions. Any other use is deemed to be improper.
- The warranty does not cover faults caused by incorrect handling, damage or attempted repairs. The same applies to normal wear and tear.

## WHEN MAY THE PRODUCT NOT BE USED?

- The product may not be used or may only be used following consultation with a doctor in the case of:
  - pregnancy,
  - slipped disc,
  - disc protrusion,
  - degenerative joint diseases,
  - diseases of the spine,
  - fractures of the spine, shoulders or ribs,
  - osteoporosis and
  - serious cardiovascular or respiratory diseases.
- If you have concerns about your health, consult a doctor BEFORE using the product.
- People who are insensitive to heat or the cold and other vulnerable people who are unable to react to overheating / undercooling may not use the gel pad.
- Do not use the product while sleeping.

## SAFETY NOTICES

### WARNING – Danger of Injury!

- Keep children and animals away from the packaging material. There is a danger of suffocation.
- Be careful when heating the gel pad in hot water! Only ever use suitable kitchen utensils such as a wooden spoon or tongs to insert it and take it out. There is a danger of scalding.
- The gel pad may be very hot after it is heated. You should therefore not touch it with your bare hands, but instead use oven gloves or a cloth.
- The gel pad must not be used if it is hotter than 60°C.
- Do not use the gel pad either hot or cold directly on the skin.
- The gel pad should not be placed on one part of the body continuously for longer than 20 minutes.
- Do not use the gel pad on body parts with no feeling.
- If pain or discomfort occurs during or after use, remove the product immediately and consult a doctor.

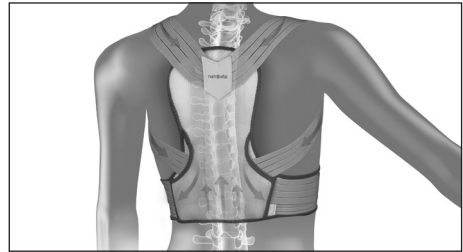
### NOTICE – Risk of Damage to Material and Property

- Protect the product from flames, persistent wet conditions or moisture, direct sunlight, extreme temperatures and sharp or pointed objects.
- Never stand on the gel pad.

## USING THE POSTURE CORRECTOR

### Please Note!

- Do not pull the straps too tight so that you do not constrict blood flow. Wearing the posture corrector should not be an unpleasant experience.
- To achieve success, the posture corrector should be worn regularly.

1. Place your arms through the straps. Make sure that the posture corrector is positioned centrally between your shoulder blades.
2. Assume a good posture: Straighten your back and stick out your chest.
3. Close the posture corrector in front of your stomach using the hook-and-loop fastener.
4. Tighten the straps so that they pull the shoulders back slightly. Fix in place with the hook-and-loop fastener.

## USING THE GEL PAD

The gel pad can be used both for cooling and for warming as you wish.

To allow it to **cool**, place the gel pad in the freezer compartment for approx. 2 hours.

If it is **to warm up**, heat the gel pad either in the microwave or in a hot water bath.

### Microwave

1. Wrap the gel pad in a damp cloth.
2. Place it in the microwave and heat at 800 W for approx. 60 seconds, at 1000 W for approx. 50 seconds or at 1200 W for approx. 40 seconds.
3. Turn over the gel pad and heat it again for the same length of time.
4. If the gel pad is not warm enough after this time, it can be heated for another 20 seconds. **However, you must not heat it for another 20 seconds more than twice.**

### Water Bath

1. Pour hot water into a bowl or a pan.
2. Carefully place in the gel pad using tongs, a wooden spoon or a similar implement.
3. After 20 – 30 seconds, use the tongs or wooden spoon to take it out and place it down on a towel.
4. If the gel pad is not yet warm enough, repeat the process.

### Use

The gel pad can be applied on its own wrapped in a cloth or together with the posture corrector.

To use it with the posture corrector, place the gel pad in the inside pocket at the back.

## CLEANING AND STORAGE

### Please Note!

- Do not use any corrosive or abrasive cleaning agents to clean the product. These may damage the surface.
- 
- Wash the posture corrector if necessary in accordance with the instructions on its washing label. Do not use fabric softener.
  - Remove any dirt from the gel pad with a damp cloth.
  - Allow the product to dry completely before storing it away.
  - Store the product in a cool and dry place protected from sunlight out of the reach of children and animals.

## PRODUCT DATA

Article number: 04053 ( Size S / M)  
04054 ( Size L / XL)

Material: 50% polyurethane, 40% polyester, 10% elastane

ID of operating instructions: Z 04053\_ 04054 M NV V1  
1120 md

## DISPOSAL

Dispose of the packaging material in an environmentally friendly manner so that it can be recycled.

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All rights reserved.
DS Produkte GmbH