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INTENDED USE

- The electrodes are to be used for stimulation applications.

APPLICATION

- Clean skin thoroughly prior to application of electrodes.
- **Attention:** Electrodes **do not stick well** if any lotions, oil, makeup, dirt, etc. is left on the skin.
- Insert the pins of the lead wire from the device into the electrode wire connectors.
- Remove the electrodes from the protective liner and apply firmly to the skin.
- The application can be improved by moistening the skin lightly with water or by using the **prorelax® electrode gel**.

REMOVAL

- Lift at the edge of electrodes and peel. Do not pull on the lead wires when removing electrodes from release liner of the skin.
- Place the electrodes on the liner and disconnect the lead wire by twisting and pulling at the electrode wire connectors.

CARE AND STORAGE

- Carefully clean the electrodes with water or a damp cloth and let them air dry briefly.
- Between uses, store the electrodes in the bag in a cool, dry place.
- The life of the electrodes varies depending on skin conditions, storage temperature and the amount of use, type of stimulation, and stimulation site. Electrodes life may be extended by carefully following the directions above or by using the **prorelax® electrode gel**.

IMPORTANT

- The life of the electrodes are intended for single patient use only.
- Do not apply to broken skin. Should a skin rash occur, discontinue use and contact your physician. Do not stimulate while driving a motorized vehicle, or operating machinery.

CONTRAINDICATION

- Electrodes must not be used for stimulation if you have a cardiac pacemaker, implanted defibrillator, or other implanted metallic or electronic device. Such use could cause electric shock, burns, electrical interference, or death.

WARNINGS

- Do not place stimulation electrodes over the neck because this could cause severemuscle spasms resulting in closure of the airway, difficulty in breathing, or adverse effects on heart rhythm or blood pressure. • Do not place stimulation electrodes across the chest because the introduction of electrical current into the chest may cause rhythm disturbances to the heart, which could be lethal. • Do not place stimulation electrodes over open wounds or rashes, or over swollen, red, infected, or inflamed areas or skin eruptions (e.g., phlebitis, thrombophlebitis, varicose veins). • Do not place stimulation electrodes over, or in proximity to, cancerous lesions. • Electrodes should be applied only to normal, intact, clean, healthy skin. • The size, shape, and type of electrodes may effect the safety and effectiveness of electrical stimulation. • The electrical performance characteristics of electrodes may effect the safety and effectiveness of electrical stimulation.
- Using stimulation electrodes that are too small or incorrectly applied could result in discomfort or skin burns. • The user should not share electrodes with other persons because of the risks of adverse skin reactions and disease transmission.

PRECAUTIONS

- The long-term effects of cutaneous electrodes for electrical stimulation and/or recording are unknown. • Since the effects of stimulation of the brain are unknown, stimulation electrodes should not be placed on opposite sides of the head. • Keep electrodes out of the reach of children. • Use caution if electrodes are applied over areas of skin that lack normal sensation. • Replace self-adhesive electrodes if they no longer stick firmly to the skin.

ADVERSE REACTIONS

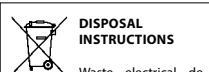
- User may experience skin irritation and burns beneath the electrodes with the use of stimulation electrodes applied to the skin. • Users may experience headache and other painful sensations during or following the application of electrical stimulation near the eyes and to the head and face.

LIMIT FOR USE

- The electrodes should not be shared.
- The electrodes should not be kept on the body longer than 1 hour for each treatment time.
- The electrodes are reusable and good for 30 times of use if kept properly cares and storage.

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DISPOSAL INSTRUCTIONS

Waste electrical devices marked with this sign must not be disposed of with your household waste, but are to be collected separately. Therefore, please return this device at the end of its life cycle to special collection points for disposal or your local dealer.