Akupressur-Set

00040

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EN Instructions for Use

Dear Customer,

We are delighted that you have chosen the MAXXMEE acupressure set. After a strenuous day, you can come home, sit back and enjoy a stimulating massage! The gentle pressure of your own body weight allows different parts of the body to be stimulated and relaxed. You may even feel a slight warming effect after lying down on the acupressure mat or acupressure pillow for a little while – this will really soothe any stress or tension you may have.

Before using the product for the first time, please read through these instructions for use carefully and keep them for future reference and other users. They form an integral part of the product. The manufacturer and importer do not accept any liability if the information in these instructions for use is not complied with.

If you have any questions about the product, contact the customer service department via our website: www.dspro.de/kundenservice

We hope you have a lot of fun with your MAXXMEE acupressure set.

Items Supplied

- 1 x mat (66x41x2 cm)
- 1 x pillow (approx. 36.5x13.5x9.5 cm)
- 1 x carrying bag
- 1 x instructions for use

Check the product for transport damage. If you find any damage, do not use the product but contact our customer service department.

Remove any possible films, stickers or transport protection from the product.

Intended Use

- This product is intended to be used for giving an acupressure massage to the body and for relaxation. It is not designed for therapeutic purposes and is not a substitute for medically required treatment. If complaints persist, consult a doctor.
- The product is designed for personal use only and is not intended for commercial applications.
- Use the product only for the specified purpose. Any other use is deemed to be improper.
- The warranty does not cover faults caused by incorrect handling, damage or attempted repairs. The same applies to normal wear and tear.

Safety Notices

- Keep children and animals away from the packaging material. There is a danger of suffocation.
- Infants and people who are sleeping or unconscious must **not** be placed on the product.
- **Only use the product if you have no physical illnesses or ailments!** If you are unsure whether it may be used, consult a doctor.
- Do not use near to or on injuries, swelling, skin irritations, infected skin, warts, fungal infections, abnormal changes in the skin or body parts with no feeling.
- Protect the product from fire and do not expose it for a prolonged period to direct sunlight, moisture or wet conditions.
- Do not use the product if it has visible signs of damage.

Use

Unroll the mat, place the pillow next to it and optionally lie, stand or sit on the mat. Thanks to the pressure of your own body weight, the nubs, each with 27 pressure spikes, deliver an acupressure effect.

Depending on how sensitive you are, you can initially place thin fabric between the skin and the nubs or wear a T-shirt. After a little while, this can be dispensed with so that the skin is in direct contact with the nubs and the acupressure effect is boosted.

Each person can determine the frequency and length of treatment for themselves. We recommend a treatment time of from 15 to 45 minutes for each section of the body.

Care and Storage

- Take the covers off the mat and the pillow. Turn the covers and the carrying bag inside out and wash them separately or with the same colours in accordance with the instructions on the washing label. Do not use any fabric softener! Allow them to dry in the air.
- Store in a dry place that is protected from direct sunlight and is out of the reach of animals.

Symbols

- | | | | |
|--|-----------------------|--|------------------------------|
| | Delicate wash at 30°C | | Do not dry in a tumble dryer |
| | Coloured wash at 30°C | | Do not iron |
| | Do not bleach | | Do not dry-clean |

Disposal

- Dispose of the packaging material in an environmentally friendly manner so that it can be recycled. The product should also be disposed of in an environmentally friendly way.

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All rights reserved.
DS Produkte GmbH, Am Heisterbusch 1, 19258 Gallin