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Dear Customer,

We are delighted that you have chosen this **ear cleaner**. If you have any questions about the product, contact the customer service department via our website: www.ds-group.de/kundenservice

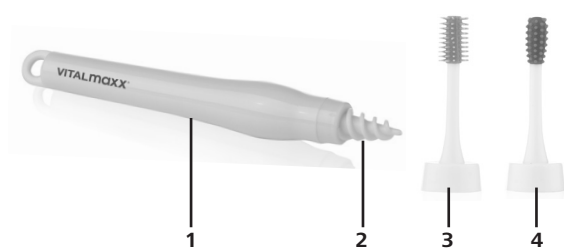
INTENDED USE

- This product is intended to be used to clean the earlobes and the external auditory canal.
- **The product is intended to be used exclusively for cleaning the user's own ears. Do not use it on others!**
- **The product is unsuitable for young children.**
- Do not use the product in the nose.
- The product is designed for personal use only and is not intended for commercial applications.
- Use the product only for the intended purpose and as described in the operating instructions. Any other use is deemed to be improper. **Incorrect use may lead to serious injuries.**
- The warranty does not cover faults caused by incorrect handling, damage, calcification or attempted repairs. The same applies to normal wear and tear.

SAFETY NOTICES

- Keep the packaging material and product away from young children and animals. There is a danger of suffocation from small parts and plastic films!
- Keep the product out of the reach of children and make sure that children do not play with the product. Incorrect use may lead to injuries.
- If you are unsure whether the product may be used, consult a doctor.
- People who have no feeling on their ears and / or hands must not use the product.
- Do not use the product in the event of injury or inflammation of the ear or auditory canal.
- For reasons of hygiene, an attachment should only be used by one person to prevent the transfer of skin infections or similar complaints.
- Do not make any modifications to the product. Use only original accessories from the manufacturer in order to guarantee that there is no interference that may prevent the product from working and to avoid possible damage.
- Protect the product from flames and heat sources.

ITEMS SUPPLIED

- 1 Handle (1 x)
 - 2 Spiral attachment (8 x) for removing larger amounts of earwax
 - 3 Bristle attachment (5 x) for smaller levels of dirt, to treat an itchy ear and to clean the earlobe
 - 4 Pimple attachment (4 x) for more stubborn earwax
- Storage box (1 x)
 - Operating instructions (1 x)

Check the items supplied for completeness and the components for transport damage. If you find any damage, do not use the product but contact our customer service department.

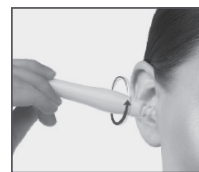
USE

Please Note!

- Before each use, check whether the product is in good condition. Damaged / worn attachments can cause injury and must be disposed of or replaced.
- Use in a quiet, secure and stationary environment to prevent any injuries caused by uncontrolled movements. Do not use in a vehicle and make sure that you cannot be bumped into by other people.
- Do not force the attachment into the ear, or insert it too deep. The eardrum could be damaged!
- If as you are using the product you experience pain or skin irritations, stop using the product immediately!

1. Select one of the attachments, plug it on the handle and twist it clockwise to secure it. If there is already an attachment on the handle, twist it anticlockwise and pull it off. Then attach the attachment you want.

2. Introduce the attachment **CAREFULLY only into the external auditory canal** and twist it to release the earwax, and then pull it out.
3. Clean the attachment thoroughly after every use (see below).

CLEANING AND STORAGE

Please Note!

- Do not use any sharp or abrasive cleaning agents or cleaning pads to clean the product. These may damage the surfaces.

- Clean the attachments with warm soapy water and rinse them with clear water.
- Wipe down the handle with a damp cloth if necessary.
- Dry all parts with a cloth or allow them to dry in the air before storing away.
- Store the product in the storage box in a cool, dry place which is protected from direct sunlight and is out of the reach of children and animals.

DISPOSAL

Dispose of the packaging material in an environmentally friendly manner so that it can be recycled.

Dispose of the product in an environmentally friendly manner as well.

DS Produkte GmbH • Stormarnring 14 • 22145 Stapelfeld • Germany

CUSTOMER SERVICE

Am Heisterbusch 1 • 19258 Gallin • Germany

☎ +49 38851 314650* • kundenservice@dspro.de

* Calls to German landlines are subject to your provider's charges.

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All rights reserved.
DS Produkte GmbH • Stormarnring 14 • 22145 Stapelfeld • Duitsland