- 🇬🇧 UNIVERSAL REPLACEMENT PADS
- 🇪🇸 PARCHES DE RECAMBIO UNIVERSAL
- 🇫🇷 ÉLECTRODES DE RECHANGE UNIVERSELLES
- 🇩🇪 UNIVERSAL-ERSATZPADS
- 🇮🇹 ELETTRODI DI RICAMBIO UNIVERSALI
- 🇳🇱 UNIVERSELE VERVANGINGSELEKTRODEN

- 🇬🇧 5 self-adhesive electrodes with snap fasteners
- 🇪🇸 5 parches autoadhesivos con corchete
- 🇫🇷 5 électrodes autoadhésives avec point de fixation
- 🇩🇪 5 selbstklebende Pads mit Klammer
- 🇮🇹 5 elettrodi adesivi con automatici
- 🇳🇱 5 zelfklevende elektroden met haakjes

The life of the pads varies depending on skin conditions, skin preparation, type of stimulation, storage and climate. After approx. 30 sessions it is advisable to replace pads for maximum results. If you find that the pads are becoming prematurely ineffective, please dampen with a wet cloth.

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International patents pending. All imitations will be prosecuted.

Gymform® Total Abs is a registered EU/CTM trade mark.

5 self-adhesive electrodes with snap fasteners
Specifically designed for the Gymform® TOTAL ABS and Gymform® TOTAL ABS PLATINUM but can be used with other types of Electro Muscular Stimulator devices.

Directions of use

- 1 – Wash and dry the skin area before applying the pads.
- 2 – Connect both the pads to the Gymform® TOTAL ABS before placing onto skin.
- 3 – Remove the protective plastic backing and place onto your desired muscle group.
- 4 – Before removing the pads from the skin always turn off the Electro Muscular Stimulator device.
- 5 – Re-place the protective plastic backing onto the Gymform® TOTAL ABS pads. This process will greatly increase the pads life span.

Warning

DO NOT apply the sticky pads to irritated skin. If a rash appears suspend use and consult your doctor. Always read the Instruction Manual before using the Electro Muscular Stimulator device.

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Best Direct International Ltd,
ISL ITALY S.r.l.
ISL GmbH