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Original Instruction Manual

3-section Portable massage table

Congratulations on the purchase of your new portable massage table. Your new table has been skillfully crafted to exacting standards, utilizing high quality components and materials. With proper use and care, as outlined in this user's guide, your table should provide a long time of reliable performance. To insure the safe and correct use of your table, please take a moment to read this entire guide prior to using your table.

Retain For Future Reference

Fabriqué pour: Made4home SAS,
137 Avenue Gustave Eiffel, 13340 Rognac, France

PRECAUTIONARY INSTRUCTIONS

- Maintain the table periodically. Make sure all screws are secure and the cable has not deteriorated or frayed. Make sure there are no cracks in the wood structure.
- Do not overload the table beyond the recommended weight limit. This could result in serious injury to the user.
- Do not use alcohol based cleaners when cleaning the vinyl. This could cause damage to the vinyl material. Use warm soapy water and wipe dry.
- Make sure when using the table that all four legs are on a solid surface and at the same level.

Table Set Up

Place the table on its side, release the buckles, partially open the table, and remove the accessories (Figure 1).

Standing behind the table, grasp both table handles and gently swing in an outward motion to fully open the table, the legs should open automatically (Figure 2).

Turn the table upright by lifting the middle of the table.

Press down in the middle of the table to make sure the table lays flat. If the table peaks in the middle, check and straighten out the cables.

Table and the Accessories Use

We provide optional accessories for you. Please read the instruction carefully before use.

Headrest:

The Headrest consists of a crescent Headrest Base and a crescent Face Pad. It will give your client comfortable head and face support. You can align the headrest base in different positions, as you can easily change the angle and the height of the headrest base by adjusting the base handle. (Figure 3)

Adjustable Armrest Shelf:

This adjustable Armrest Shelf is a good device to support your client's arms at the front of the table. Hang the Armrest Shelf on the two bars of the headrest and slide the belt up or down for the optimum position. (Figure 4).

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