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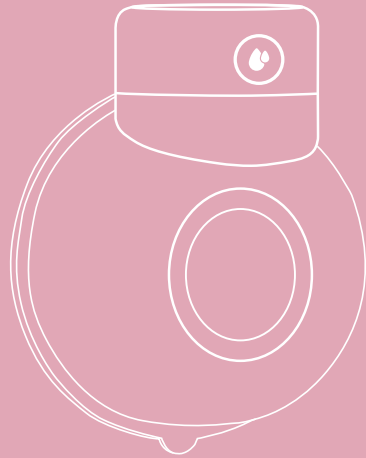
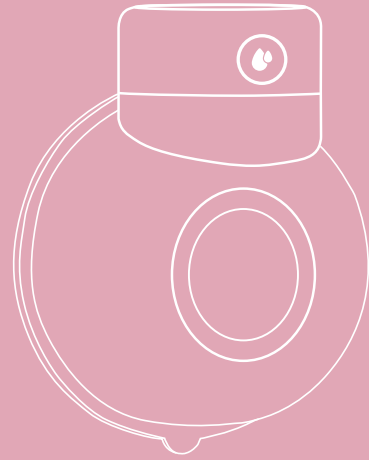
09


More Light More Silent More Comfortable

S12-USER MANUAL

Read this user manual carefully before using your electric breast pump, and save the user manual for future reference.

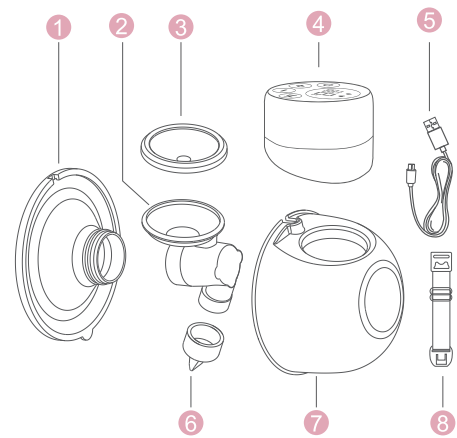Overview


Parts list

- | | |
|-------------------------|----------------------------|
| ① Silicone shield x1 | ② Linker x1 |
| ③ Silicone diaphragm x1 | ④ Pump motor x1 |
| ⑤ type-c cable x1 | ⑥ Valve x1 |
| ⑦ Milk collector x1 | ⑧ Bra adjustment buckle x1 |

01

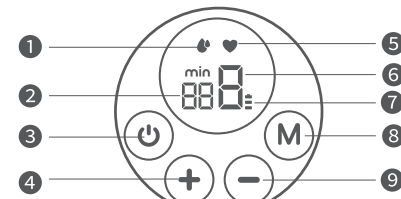Control panel

- | | |
|---------------------------|-------------------------|
| ① Expression mode display | ② Timing display |
| ③ Power button | ④ Level up button |
| ⑤ Massage mode display | ⑥ Level display |
| ⑦ Battery power display | ⑧ Mode selection button |
| ⑨ Level down button | |

Note

This product has a built-in lithium battery. When the battery indicator flashes in red, we recommend that you use a certified 5V=1A adapter to charge the pump motor as soon as possible.

Cleaning and sterilizing

- For the first time and before each use, you must disassemble and wash and sterilize all the breast pump parts that come into contact with breast milk.

02

- Drinking water recommended for cleaning.
- Disassemble and clean the parts in contact with the breast and breast milk immediately after use, to avoid residual breast milk that may cause bacteria growing.

Warning

- Never wash or sterilize the pump motor and USB cable. These components do not come into with your breast milk. You can wipe those parts clean with a soft cloth.
- Take care when cleaning the valve and the silicone diaphragm. If it gets damaged, your breast pump will not function properly.
- To clean the valve, wash it gently between your fingers in warm soapy water. Do not insert objects as this may cause damage.

Assembling

- Put the silicone diaphragm into the linker, and press it with your hands to make the edges fasten tightly to ensure a perfect seal. (Figure 1)
- Install the valve into the bottom of the linker, make sure it is installed in the correct direction. (Figure 2)

Figure 1



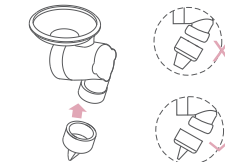
Figure 2

03

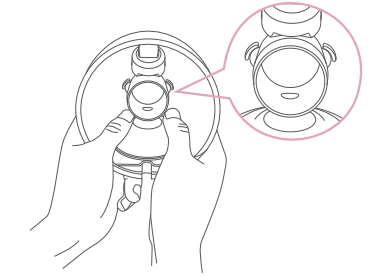- Put the assembled linker into the milk collector, with the "ear" part of the linker aligned with the internal slot of the milk collector, and press the linker disc part with both hands to fasten the edges firmly to ensure complete sealing.

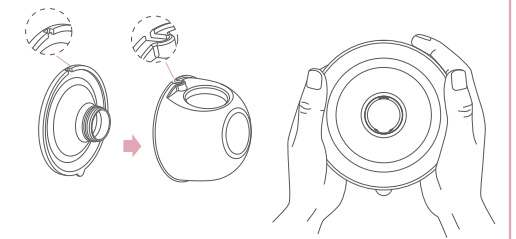
- Install the silicone shield on the milk collector, align the deflector on the horn with the deflector on the milk collector, align the groove on silicone shield in milk collector groove position.


04

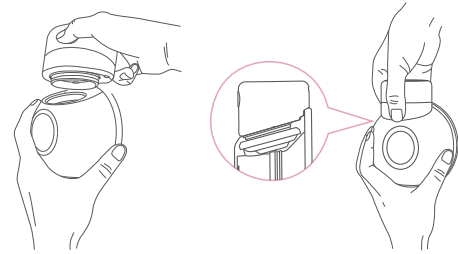- Press the pump motor onto the milk collector ensure that it fits perfectly.

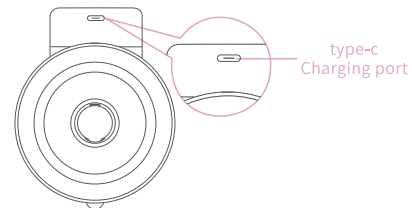- Unplug the dust plug and connect the type-c cable to charge the pump motor.


Note

The power adapter is a common standard accessory on the market. This product is not provided and can be purchased separately if necessary. Power adapter specification: 5V=1A.

05

TO begin pumping

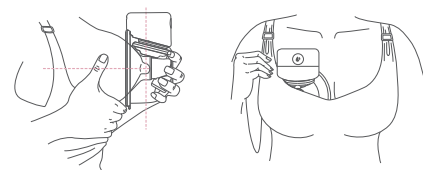- Press the silicone shield firmly against the breast, leave no gaps and make sure your nipples are centered. Keep it upright. Be careful not to distort the position of the product and the chest, otherwise the breast pump will not have suction.


- Lightly press the power button to start the breast pump, and the timing display turns on.


06

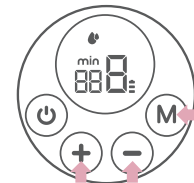- Lightly press the "M" key to switch the mode, and the display shows "♥" for the breast pumping mode. There are 9 levels in this mode. You can select the suction power that makes you comfortable by touching the "+" and "-" keys. During postpartum breastfeeding, breast discomfort such as swelling and lumps are prone to occur. Using this mode can alleviate breast problems.


- Press the "M" key again to switch to the massage mode, and the display shows "♥" for the massage mode. There are 9 levels in this mode. Each level have 10 short suction and short release plus 1 long suction and long release as a cycle. You can press the "+" and "-" keys to select the suction power that makes you comfortable. In the early postpartum period, the breast duct is not easy to conduct, and the breast can be stimulated by changing the mode to promote breast milk secretion.

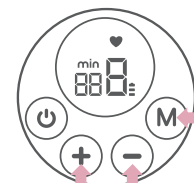07

Tips: The machine has a memory function. When you turn it on again, the machine will automatically enter the working mode and suction position you used last time.

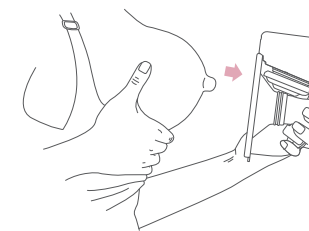- When you have finished pumping, press the power button to turn off the breast pump, then carefully remove the breast pump from breast.

08

Tips

Breast milk storage

- Store only breast milk collected by a sterilized breast pump.
- Milk can be stored in a clean and sterilized milk storage bag or sealed bottle and immediately refrigerated or frozen.

Breast milk refrigeration

- Do not freeze breast milk repeatedly.
- Do not mix fresh breast milk with frozen breast milk.
- Be sure to discard the breast milk left over after feeding.
- The extracted milk can be refrigerated in the refrigerator for 48 hours (inside the refrigerator), or frozen in the refrigerator for 3 months. Put a label on the milk bottle or milk bag, clearly indicating the date and time of the milk pump. Eat the milk that is first expressed.

Breast milk thawing

Put the frozen breast milk in the refrigerator compartment for one night to thaw. In an emergency, you can use a bowl of hot water to thaw breast milk. After thawing breast milk, it must be consumed within 24 hours.

Breast milk thawing

- Put the baby bottle upright in a bowl of hot water or warmer to heat it quickly.
- Remember! Before feeding your baby, be sure to check the temperature of breast milk.
- Do not heat breast milk in the microwave, as this may destroy important nutrients and antibodies.
- In addition, uneven heating may cause the liquid to overheat locally and may therefore cause burns.
- Do not immerse the milk storage bottle in boiling water for heating, to avoid crack or split.

09