Nasivent Sport - More Oxygen - More Power

The **Nasivent Sport Tubes** help you to improve nasal breathing. An adequate supply of oxygen plays a key role in physical exertion and endurance sports. It is therefore important that you can breathe freely.

Nasivent Sport is easy to insert into the nose and expands the nostrils in a pleasant way. It is flexible, soft and skin-friendly. Thanks to the 5 different sizes, the Nasivent Sport Tubes adapt perfectly to different nose shapes. Thanks to the retaining pins optimised for sports, these always have a firm fit in the nose, even under higher loads.

Areas of application:

To optimise oxygen supply during sports; for blocked nose and as an alternative to nasal spray; for use as a remedy for snoring

Material:

High quality silicone; BPA-free.

Safety note: Do not cut the connecting bar of the tubes! Not suitable for children under 6 years of age.

Clean the Nasivent Tubes with soap and water before use.

Insert the Nasivent Tubes completely into the nose, making sure that the retaining pins point upwards.

If the Nasivent tubes are too long, you can simply shorten them with scissors.

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